Manual Wheelchair
Silla de ruedas manual
Cadeira de Rodas Manual

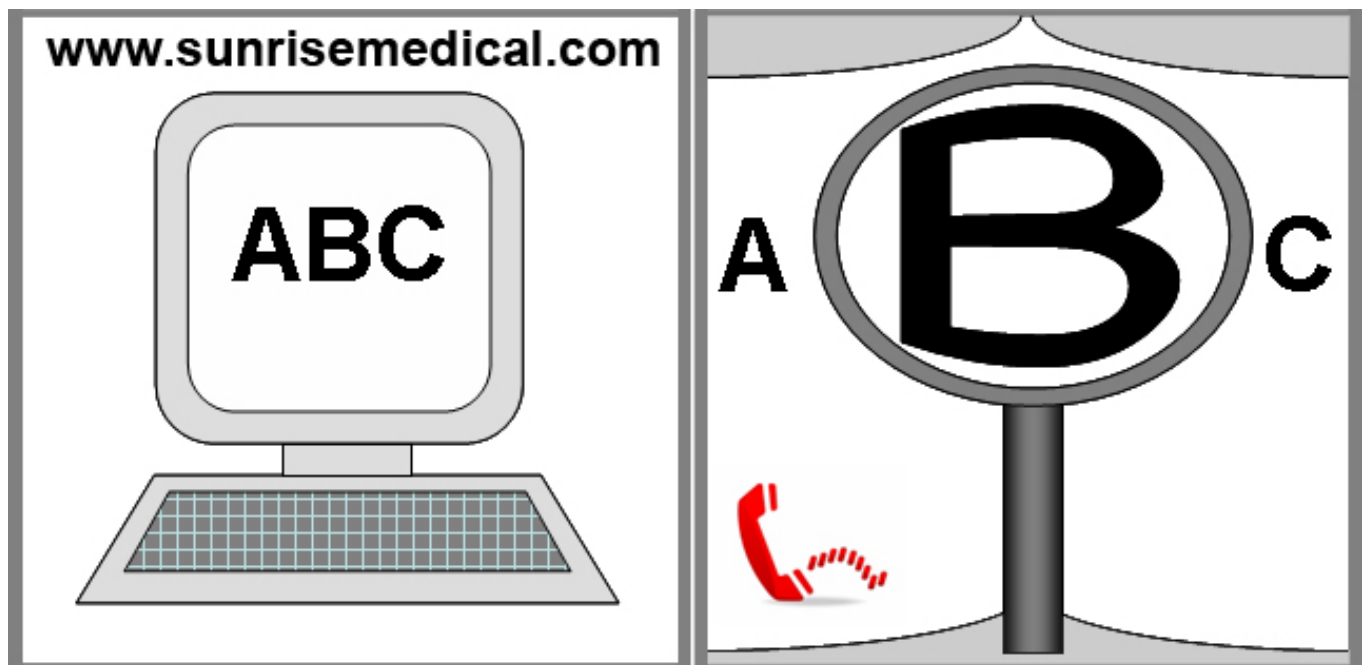
Zippie IRIS

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101510

**IF YOU ARE VISUALLY IMPAIRED, THIS DOCUMENT CAN BE VIEWED IN PDF
FORMAT AT WWW.SUNRISEMEDICAL.CO.UK**

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Wheelchair Parts:

1. Backpost
2. Height Adjustable Armrest
3. Wheel Locks
4. Aluminium Foot Plate
5. Push Handle
6. Tilt in Space Release Lever
7. 12 inch Rear Wheel
8. Carriage
9. Castor Fork
10. Castor Tyre
11. Tilt Indicator
12. Adjustable Seat Pan
13. Rocker Arm
14. Hanger Receiver
15. Seat Rail
16. Adjustable Stroller Handle Hinge

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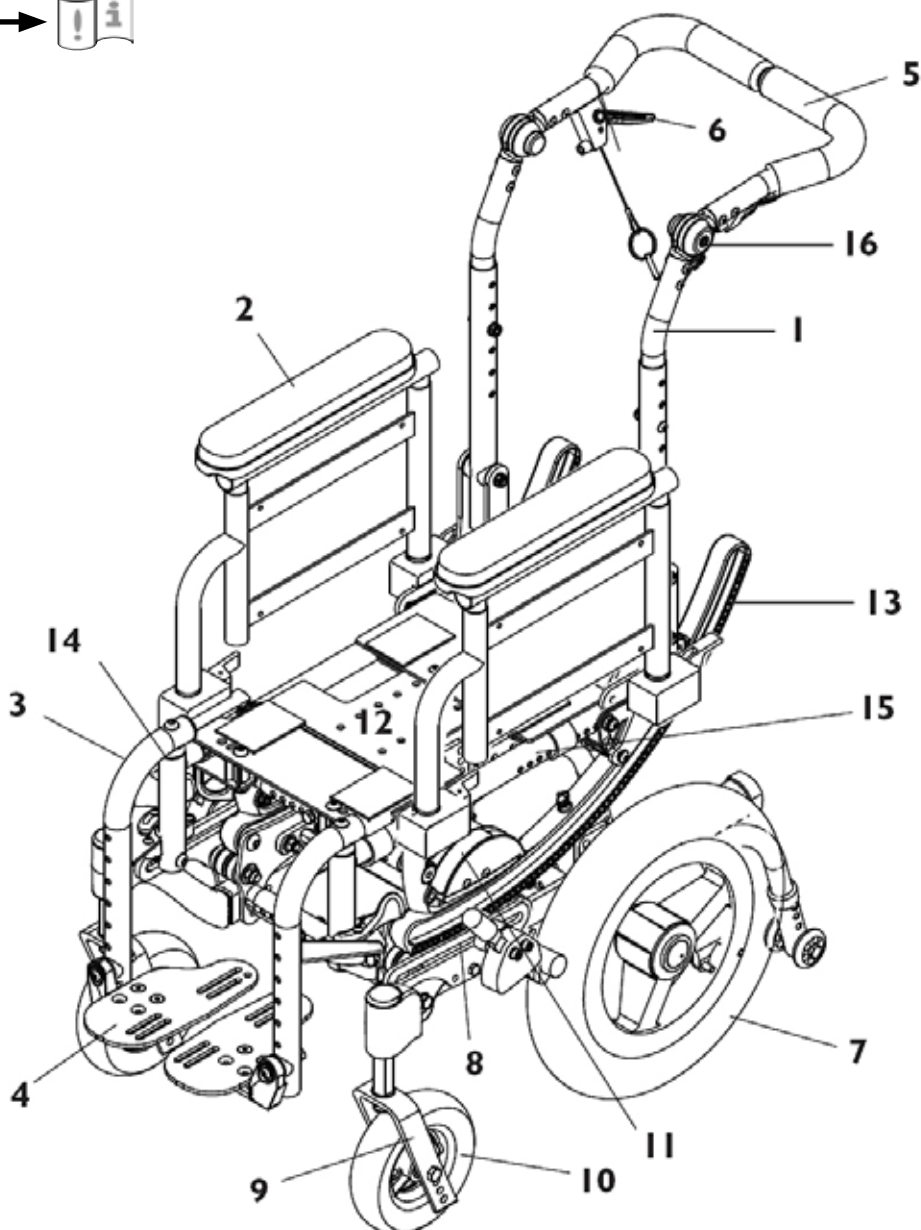
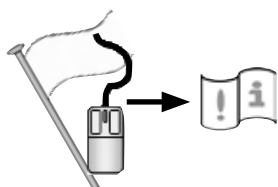
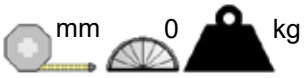
Table of Contents

Table of Contents 5
 Definitions 5
 Foreword 6
 Use 6
 Area of Application. 6
 General safety notes, warnings and driving restrictions 7
 General Warnings 8
 Warnings: Falls and Tip-Overs 9
 Warnings: Safe Use (Attendants) 12
 Warnings: Components and Options 13
 Set-up and Adjustment 16
 Battery Tray 27
 Positioning Belt 29
 Transportability 30
 Troubleshooting 32
 Maintenance and care 33
 Name Plate 34
 Warranty 35
 Disposal / recycling of materials 35

Definitions

3.1 Definitions of words used in this manual

Word	Definition
 DANGER!	Advice to the user of Potential Risk of serious injury or death if the advice is not followed
 WARNING!	Advice to the user of a potential risk of injury if the advice is not followed
 CAUTION!	Advice to user that potential damage to equipment may occur if the advice is not followed
NOTE:	General advice or best practice
	Reference To Additional Documentation

NOTE:
 The wheelchairs shown and described in this user guide may not correspond in every detail exactly to your own model. However, all instructions are completely relevant, regardless of possible detail differences.
 The manufacturer reserves the right to alter without notice any weights, measurements or other technical data shown in this manual. All figures, measurements and capacities shown in this manual are approximate, and do not constitute specifications.

NOTE:
 Please keep a note of your local service agent's address and telephone number in the space provided.
 In the event of a breakdown, contact them and try to give all relevant details so they can help you quickly.

Dealer signature and stamp

Foreword

Dear Customer,

We are very pleased that you have decided in favour of a high-quality SUNRISE MEDICAL product.

This Owner's manual will provide numerous tips and ideas so that your new wheelchair can become a trustworthy and reliable partner in your life.

Maintaining close ties to our customers means a lot to us at Sunrise Medical. We like to keep you up-to-date about new and current developments at our company. Keeping close to our customers means: fast service, as little red tape as possible, working closely with customers. When you need replacement parts or accessories, or if you just have a question about your wheelchair – we are there for you.

We want you to be satisfied with our products and service. At Sunrise Medical we are constantly working to develop our products further. For this reason, changes can occur in our palette of products with regard to form, technology, and equipment. Consequently, no claims can be construed from the data or pictures contained in this owner's manual.

The management system of SUNRISE MEDICAL is certified to DIN EN ISO 9001, ISO 13485 and ISO 14001.

As the manufacturer, SUNRISE MEDICAL, declares that the lightweight wheelchairs conform to the 93/42/EEC guideline.

NOTE:

General user advice.

Not following these instructions may result in physical injury, damage to the product or damage to the environment!

A notice to the user and/or patient that any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

B4Me special adaptations

Sunrise Medical strongly recommends that in order to ensure that your B4Me product operates, and performs as intended by the manufacturer; all the user information supplied with your B4Me product is read and understood, before the product is first used.

Sunrise Medical also recommends that the user information is not discarded after reading it, but it is kept safely stored for future reference.

Medical Device Combinations

It may be possible to combine this Medical device with one or more other Medical Device or other product. Information on which combinations are possible can be found at www.Sunrisemedical.co.uk. All combinations listed have been validated to meet the General Safety and Performance Requirements, section 14.1 of the Medical Device Regulation 2017/745.

Guidance on the combination, such as mounting, can be found at www.SunriseMedical.co.uk.

Please contact your local, authorised SUNRISE MEDICAL dealer if you have any questions regarding the use, maintenance or safety of your wheelchair.

In case there is no authorised dealer in your area, or you have any questions, contact Sunrise Medical either in writing or by telephone.

Sunrise Medical
Thorns Road
Brierley Hill
West Midlands
DY5 2LD
England
Phone: 0845 605 66 88
Fax: 0845 605 66 89
www.SunriseMedical.co.uk

IMPORTANT:

DO NOT USE YOUR WHEELCHAIR UNTIL THIS MANUAL HAS BEEN READ AND UNDERSTOOD.

This symbol means Medical Device

Use

Wheelchairs are exclusively for a user who is unable to walk or has limited mobility, for their own personal use, when self-propelling and being moved by a third party (pushed by attendant) within the home and outdoors.

The maximum weight limit (includes both the user and any weight of accessories fitted to the wheelchair) is marked on the serial number label, which is affixed to the cross-brace or the stabilising bar under the seat.

Warranty can only be issued if the product is used under the specified conditions and for the intended purpose. The expected life of the wheelchair is 5 years. NO NON-STANDARD PARTS are to be fitted, other than those officially approved by Sunrise Medical.

Area of Application.

The variety of options as well as the modular design means that it can be used by those who cannot walk or have limited mobility because of:

- Paralysis
- Loss of extremity (leg amputation)
- Extremity defect/deformity
- Joint contractures/joint injuries
- Illnesses such as heart and circulation deficiencies, disturbance of equilibrium or cachexia as well as for elderly people who still have strength in the upper body.

When considering provision, please also note the body size, weight, physical and psychological constitution, the age of the person, living conditions and environment.

The engineering and construction of this wheelchair has been designed to provide maximum safety. International safety standards currently in force have either been fulfilled or exceeded in parts. Nevertheless, users may put themselves at risk by improperly using their wheelchairs. For your own safety, the following rules must be strictly observed.

Unprofessional or erroneous changes or adjustments increase the risk of accident. As a wheelchair user, you are also part of the daily traffic on streets and pavements, just like anyone else. We would like to remind you that you are therefore also subject to any and all traffic laws.

Be careful during your first ride in this wheelchair. Get to know your wheelchair.

Before each use, the following should be checked:

- Quick-release axles on the rear wheels
- Velcro on seats and backrests
- Tyres, tyre pressure and wheel locks.

Before changing any of the adjustments of this wheelchair, it is important to read the corresponding section of the user's manual.

It is possible that potholes or uneven ground could cause this wheelchair to tip over, especially when riding uphill or downhill. When riding over a step or up an incline frontally, the body should be leaning forward.

CHOOSE THE RIGHT CHAIR & SAFETY OPTIONS

- Sunrise provides a choice of many wheelchair styles to meet the needs of the wheelchair rider. However, final selection of the type of wheelchair, options and adjustments rests solely with you and your health care advisor. Choosing the best chair and set-up for your safety depends on such things as:
 1. Your disability, strength, balance and coordination.
 2. The types of hazards you must overcome in daily use (where you live and work, and other places you are likely to use your chair).
 3. The need for options for your safety and comfort (such as anti-tip tubes, positioning belts, or special seating systems).

REVIEW THIS MANUAL OFTEN

Before using this chair you, and each person who may assist you, should read this enTyre manual and make sure to follow all instructions. Review the warnings often, until they are second nature to you

WARNINGS GROUPS

The word "WARNING" refers to a hazard or unsafe practice that may cause severe injury or death to you or to other persons. The "Warnings" are in four main sections, as follows:

GENERAL WARNINGS

Here you will find a safety checklist and a summary of risks you need to be aware of before you ride this chair.

WARNINGS– FALLS & TIP-OVERS

Here you will learn how to avoid a fall or tip-over while you perform daily activities in your chair.

WARNINGS– FOR SAFE USE

Here you will learn about practices for the safe use of your chair.

WARNINGS– COMPONENTS & OPTIONS

Here you will learn about the components of your chair and options you can select for safety. Consult your authorized supplier and your health care advisor to help you choose the best set-up and options for safe use.

NOTE: Where they apply, you will also find "Warnings" in other sections of this manual. Heed all warnings in this section. If you fail to do so a fall, tip-over or loss of control may occur and cause severe injury to the rider or others

General Warnings

WARNINGS!

WEIGHT LIMIT

NEVER exceed the weight limit of 75kg, (165.5 lbs.) for the Zippie IRIS, for combined weight of rider and items carried. If you do exceed the limit, damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

WEIGHT TRAINING

NEVER use this chair for weight training if total weight (rider plus equipment) exceeds 68kg, (150 lbs.) If you do exceed the limit, damage to your chair, a fall, tipover or loss of control may occur and cause severe injury to the rider or others.

GETTING TO KNOW YOUR CHAIR

Every wheelchair is different. Take the time to learn the feel of this chair before you begin riding. Start slowly, with easy, smooth strokes. If you are used to a different chair, you may use too much force and tip over. If you use too much force, damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

TO REDUCE THE RISK OF AN ACCIDENT

1. BEFORE riding, you should be trained in the safe use of this chair by your health care advisor.
2. Practice bending, reaching and transfers until you know the limit of your ability. Have someone help you until you know what can cause a fall or tip-over and how to avoid doing so.
3. Be aware that you must develop your own methods for safe use best suited to your level of function and ability.
4. NEVER try a new maneuver on your own until you are sure you can do it safely.
5. Get to know the areas where you plan to use your chair. Look for hazards and learn how to avoid them.
6. Use anti-tip tubes unless you are a skilled rider of this chair and are sure you are not at risk to tip over. If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

SAFETY CHECKLIST

Before Each Use Of Your Chair:

1. Make sure the chair rolls easily and that all parts work smoothly. Check for noise, vibration, or a change in ease of use. (They may indicate low Tyre pressure, loose fasteners, or damage to your chair).
2. Repair any problem. Your authorized supplier can help you find and correct the problem.
3. Check to see that both quick-release rear axles are locked. When locked, the axle button will "pop out" fully. If not locked, the wheel may come off and cause you to fall.
4. If your chair has anti-tip tubes, lock them in place.

NOTE: See "Anti-Tip Tubes" (Section VIII) for times you should not use them.

General Warnings

DANGER!

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

CHANGES & ADJUSTMENTS

1. If you modify or adjust this chair, it may increase the risk of a tip-over UNLESS you make other changes as well.
2. Consult your authorized supplier BEFORE you modify or adjust your chair.
3. We recommend that you use anti-tip tubes until you adapt to the change, and are sure you are not at risk to tip over.
4. Unauthorized modifications or use of parts not supplied or approved by Sunrise may change the chair structure. This will void the warranty and may cause a safety hazard.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

ENVIRONMENTAL CONDITIONS

1. Use extra care if you must ride your chair on a wet or slick surface. If you are in doubt, ask for help.
2. Contact with water or excess moisture may cause your chair to rust or corrode. This could cause your chair to fail.
 - a. Do not use your chair in a shower, pool or other body of water. The chair tubing and parts are not water-tight and may rust or corrode from the inside.
 - b. Avoid excess moisture (for example, do not leave your chair in a damp bathroom while taking a shower).
 - c. Dry your chair as soon as you can if it gets wet, or if you use water to clean it.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

TERRAIN

1. Your chair is designed for use on firm, even surfaces such as concrete, asphalt and indoor floors and carpeting.
2. Do not operate your chair in sand, loose soil or over rough terrain. This may damage wheels or axles, or loosen fasteners of your chair.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

STREET USE

In most states, wheelchairs are not legal for use on public roads. Be alert to the danger of motor vehicles on roads or in parking lots.

1. At night, or when lighting is poor, use reflective tape on your chair and clothing.
2. Due to your low position, it may be hard for drivers to see you. Make eye contact with drivers before you go forward. When in doubt, yield until you are sure it is safe.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

General Warnings

MOTOR VEHICLE SAFETY

As indicated on the front cover of this user manual, identify whether your chair has been manufactured with the Transit Option installed.

WARNING!

If your chair is not equipped with Transit Option, this wheelchair does not meet federal standards for motor vehicle seating.

1. NEVER let anyone sit in this chair while in a moving vehicle.
2. ALWAYS secure the rider with proper motor vehicle restraints. In an accident or sudden stop the rider may be thrown from the chair. Wheelchair seat belts will not prevent this, and further injury may result from the belts or straps.
3. NEVER transport this chair in the front seat of a vehicle. It may shift and interfere with the driver.
4. ALWAYS secure this chair so that it cannot roll or shift.
5. Do not use any chair that has been involved in a motor vehicle accident.

If your chair is equipped with the Transit Option: See section VIII, Part N: Transit Option and section VII, Part G: Transit Use.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

WHEN YOU NEED HELP

For The Rider:

Make sure that each person who helps you reads and follows all warnings and instructions that apply.

For Attendants:

1. Work with the rider's doctor, nurse or therapist to learn safe methods best suited to your abilities and those of the rider.
2. Tell the rider what you plan to do, and explain what you expect the rider to do. This will put the rider at ease and reduce the risk of an accident.
3. Make sure the chair has push handles. They provide secure points for you to hold the rear of the chair to prevent a fall or tip-over. Check to make sure push handle grips will not rotate or slip off.
4. To prevent injury to your back, use good posture and proper body mechanics. When you lift or support the rider or tilt the chair, bend your knees slightly and keep your back as upright and straight as you can.
5. Remind the rider to lean back when you tilt the chair backward.
6. When you descend a curb or single step, slowly lower the chair in one easy movement. Do not let the chair drop the last few inches to the ground. This may damage the chair or injure the rider.
7. To avoid tripping, unlock and rotate anti-tip tubes up, out of the way. ALWAYS lock the rear wheels and lock anti-tip tubes in place if you must leave the rider alone, even for a moment. This will reduce the risk of a tip over or loss of control of the chair.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

Warnings: Falls and Tip-Overs

WARNINGS!

CENTER OF GRAVITY

The point where this chair will tip forward, back, or to the side depends on its center of gravity and stability. How your chair is set up, the options you select and the changes you make may affect the risk of a fall or tip-over.

1. The Most Important Adjustments Are:
 - a. The position of the rear wheels. The more you move the rear wheels forward, the more likely your chair will tip over backward.
 - b. The position of the carriage.
2. The Center Of Gravity Is Also Affected By:
 - a. A change in the set-up of your chair, including:
 - The seat height and seat angle.
 - Backrest angle.
 - b. A change in your body position, posture or weight distribution.
 - c. Riding your chair on a ramp or slope.
 - d. The use of a back pack or other options, and the amount of added weight.
3. To Reduce The Risk Of An Accident:
 - a. Consult your doctor, nurse or therapist to find out what axle and caster position is best for you.
 - b. Consult your authorized supplier BEFORE you modify or adjust this chair. Be aware that you may need to make other changes to correct the center of balance.
 - c. Have someone help you until you know the balance points of your chair and how to avoid a tip-over.
 - d. Use anti-tip tubes.
 - e. Never allow the presence of a second occupant climbing or standing on a component of the chair.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

DRESSING OR CHANGING CLOTHES

Your weight may shift if you dress or change clothes while seated in this chair.

To reduce the risk of a fall or tip-over:

1. Rotate the front casters until they are as far forward as possible. This makes the chair more stable.
2. Lock anti-tip tubes in place. (If your chair does not have anti-tip tubes, back it up against a wall and lock both rear wheels).

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

WHEELIES

Because of the ability to adjust the center of gravity and changes in center of gravity that will result from operating the tilt in space feature of this chair, as a user, do not perform wheelies in the IRIS wheelchair. Doing a "wheelie" means: balancing on the rear wheels of your chair, while the front casters are in the air. It is dangerous to do a "wheelie" as a fall or tip-over may occur.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

Warnings: Falls and Tip-Overs **WARNINGS!****OBSTACLES**

Obstacles and road hazards (such as potholes and broken pavement) can damage your chair and may cause a fall, tip-over or loss of control.

To avoid these risks:

1. Keep a lookout for danger – scan the area well ahead of your chair as you ride.
2. Make sure the floor areas where you live and work are level and free of obstacles.
3. Remove or cover threshold strips between rooms.
4. Install a ramp at entry or exit doors. Make sure there is not a drop off at the bottom of the ramp.
5. To Help Correct Your Center Of Balance:
 - a. Lean your upper body FORWARD slightly as you go UP over an obstacle.
 - b. Press your upper body BACKWARD as you go DOWN from a higher to a lower level.
6. If your chair has anti-tip tubes, lock them in place before you go UP over an obstacle.
7. Keep both of your hands on the handrims as you go over an obstacle.
8. Never push or pull on an object (such as furniture or a doorjamb) to propel your chair.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

REACHING OR LEANING

If you reach or lean it will affect the center of balance of your chair. This may cause you to fall or tip over. When in doubt, ask for help or use a device to extend your reach.

1. NEVER reach or lean if you must shift your weight sideways or rise up off the seat.
2. NEVER reach or lean if you must move forward in your seat to do so. Always keep your buttocks in contact with the backrest.
3. NEVER reach with both hands (you may not be able to catch yourself to prevent a fall if the chair tips).
4. NEVER reach or lean to the rear unless your chair has anti-tip tubes locked in place.
5. DO NOT reach or lean over the top of the seat back. This may damage one or both backrest tubes and cause you to fall.
6. If You Must Reach Or Lean:
 - a. Do not lock the rear wheels. This creates a tip point and makes a fall or tip-over more likely.
 - b. Do not put pressure on the footrests.

NOTE: Leaning forward puts pressure on the footrests and may cause the chair to tip if you lean too far.

- c. Move your chair as close as you can to the object you wish to reach.
- d. Do not try to pick up an object from the floor by reaching down between your knees. You are less likely to tip if you reach to the side of your chair.
- e. Rotate the front casters until they are as far forward as possible. This makes the chair more stable.

NOTE: To do this: Move your chair past the object you want to reach, then back up alongside it. Backing up will rotate the casters forward.

- f. Firmly grasp a rear wheel or an armrest with one hand. This will help to prevent a fall if the chair tips.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

Warnings: Falls and Tip-Overs **WARNINGS!****MOVING BACKWARD**

Use extra care when you move your chair backward. Your chair is most stable when you propel yourself forward. You may lose control or tip over if one of the rear wheels hits an object and stops rolling.

1. Propel your chair slowly and smoothly.
2. If your chair has anti-tip tubes, make sure to lock them in place.
3. Stop often and check to be sure your path is clear.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

RAMPS, SLOPES & SIDEHILLS

Riding on a slope, which includes a ramp or sidehill, will change the center of balance of your chair. Your chair is less stable when it is at an angle. Anti-tip tubes may not prevent a fall or tip-over.

1. Do not use your chair on a slope steeper than 10%. (A 10% slope means: one foot in elevation for every ten feet of slope length.)
2. Always go as straight up and as straight down as you can. (Do not “cut the corner” on a slope or ramp.)
3. Do not turn or change direction on a slope.
4. Always stay in the CENTER of the ramp. Make sure ramp is wide enough that you are not at risk that a wheel may fall over the edge.
5. Do not stop on a steep slope. If you stop, you may lose control of your chair.
6. NEVER use rear wheel locks to try to slow or stop your chair. This is likely to cause your chair to veer out of control.
7. Beware Of:
 - a. Wet or slippery surfaces.
 - b. A change in grade on a slope (or a lip, bump or depression). These may cause a fall or tip-over.
 - c. A drop-off at the bottom of a slope. A drop-off of as small as 3/4 inch can stop a front caster and cause the chair to tip forward.
8. To Reduce The Risk Of A Fall Or Tip-Over:
 - a. Lean or press your body UPHILL. This will help adjust for the change in the center of balance caused by the slope or sidehill.
 - b. Keep pressure on the handrims to control your speed on a down slope. If you go too fast you may lose control.
 - c. Ask for help any time you are in doubt.
9. Ramps At Home & Work - For your safety, ramps at home and work must meet all legal requirements for your area. We recommend:
 - a. Width: At least four feet wide.
 - b. Guardrails: To reduce the risk of a fall, sides of ramp must have guardrails (or raised borders at least three inches high).
 - c. Slope: Not more than a 10% grade.
 - d. Surface: Flat and even, with a thin carpet or other non-skid material. (Make sure there is no lip, bump or depression.)
 - e. Bracing: Ramps must be STURDY. You may need bracing so ramps do not “bow” when you ride on them.
 - f. Avoid A Drop Off: You may need a section at the top or bottom to smooth out the transition.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

 **WARNINGS!****TRANSFER**

It is dangerous to transfer on your own. It requires good balance and agility. Be aware that there is a point during every transfer when the wheelchair seat is not below you.

To Avoid A Fall:

1. Work with your health care advisor to learn safe methods.
 - a. Learn how to position your body and how to support yourself during a transfer.
 - b. Have someone help you until you know how to do a safe transfer on your own.
2. Lock the rear wheels before you transfer. This keeps the rear wheels from rolling.

NOTE: This will NOT keep your chair from sliding away from you or tipping.

3. Make sure to keep pneumatic Tyres properly inflated. Low Tyre pressure may allow the rear wheel locks to slip.
4. Move your chair as close as you can to the seat you are transferring to. If possible, use a transfer board.
5. Rotate the front casters until they are as far forward as possible.
6. If you can, remove or swing footrests out of the way.
 - a. Make sure your feet do not catch in the space between the footrests.
 - b. Avoid putting weight on the footrests as this may cause the chair to tip.
7. Make sure armrests are out of the way and do not interfere.
8. Transfer as far back onto the seat surface as you can. This will reduce the risk that the chair will tip or move away from you.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

CURBS & STEPS

1. Each person who helps you should read and follow the warnings "For Safe Use" (Section VII) and "For Attendants" (Section V, Part K).
2. Do not try to climb or descend a curb or step alone.
3. Unlock and rotate anti-tip tubes up, out of the way, so they do not interfere.
4. Do not try to climb a high curb or step UNLESS you have help. Doing so may cause your chair to exceed its balance point and tip over.
5. Go straight up and straight down a curb or step. If you climb or descend at an angle, a fall or tip-over is likely.
6. Be aware that the impact of dropping down from a curb or step can damage your chair or loosen fasteners.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

 **WARNINGS!****STAIRS**

1. NEVER use this chair on stairs UNLESS you have someone to help you. Doing so is likely to cause a fall or tip-over.

2. Persons who help you should read and follow the warnings "For Attendants" (Section V, Part K), and "For Safe Use" (Section VII).

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

ESCALATORS

NEVER use this chair on an escalator, even with an attendant. If you do, a fall or tip-over is likely.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

Warnings: Safe Use (Attendants) **WARNINGS!**

Before you assist a rider, be sure to read the warnings “For Attendants” in Section V, Part K and follow all instructions that apply. Be aware that you will need to learn safe methods best suited to your abilities.

TILT USE

In preparation to operate the tilt:

- Always verify that the arms of the rider are stable on the armrests or within the armrests.
- Always verify that the legs of the rider are stable on the footrest/legrest.
- Never place hands, feet or foreign objects into the tilt mechanism.
- Never push or propel the chair with the seat tilted in a forward position.
- Never add chair accessories that are not specifically designed for Zippie IRIS.
- Never exceed a tilted position that aligns the backrest parallel to the ground.

DESCENDING A CURB OR SINGLE STEP

Follow these steps to help a rider descend a curb or single step going **BACKWARD**:

1. Stay at the rear of the chair.
2. Several feet before you reach the edge of the curb or step, turn the chair around and pull it backward.
3. While looking over your shoulder, carefully step back until you are off the curb or stair and standing on the lower level.
4. Pull the chair toward you until the rear wheels reach the edge of the curb or step. Then allow the rear wheels to slowly roll down onto the lower level.
5. When the rear wheels are safely on the lower level, tilt the chair back to its balance point. This will lift the front casters off the curb or step.
6. Keep the chair in balance and take small steps backward. Turn the chair around and gently lower front casters to the ground.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

CLIMBING A CURB OR SINGLE STEP

Follow these steps to help the rider climb a curb or single step going **FORWARD**:

1. Stay behind the chair.
2. Face the curb and tilt the chair up on the rear wheels so that the front casters clear the curb or step.
3. Move forward, placing the front casters on the upper level as soon as you are sure they are past the edge.
4. Continue forward until the rear wheels contact the face of the curb or step. Lift and roll the rear wheels to the upper level.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

Warnings: Safe Use (Attendants) **WARNINGS!****CLIMBING STAIRS**

1. Use at least two attendants to move a chair and rider up stairs.
2. Move the chair and rider **BACKWARD** up the stairs.
3. The person at the rear is in control. He or she tilts the chair back to its balance point.
4. A second attendant at the front should firmly grasp the hanger receiver or frontmost part of the seat rail and lift the chair up and over one stair at a time.
5. The attendants move to the next stair up. Repeat for each stair, until you reach the landing.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

DESCENDING STAIRS

1. Use at least two attendants to move a chair and rider down stairs.
2. Move the chair and rider **FORWARD** down the stairs.
3. The person at the rear is in control. He or she tilts the chair to the balance point of the rear wheels and rolls it to the edge of the top step.
4. A second attendant stands on the third step from the top and grasps the hanger receiver or frontmost part of the seat rail. He or she lowers the chair one step at a time by letting the rear wheels roll over the stair edge.
5. The attendants move to the next stair down. Repeat for each stair, until you reach the landing.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

MAINTENANCE

1. Inspect and maintain this chair strictly per chart in section XI. Maintenance.
2. If you detect a problem, make sure to service or repair the chair before use.
3. At least once a year, have a complete inspection, safety check and service of your chair made by an authorized supplier.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others. For additional maintenance information see Section XI: Maintenance.

⚠️ WARNINGS!

TRANSIT USE

Refer also to section VIII, part N and section V, part J for additional transit option and motor vehicle warnings.

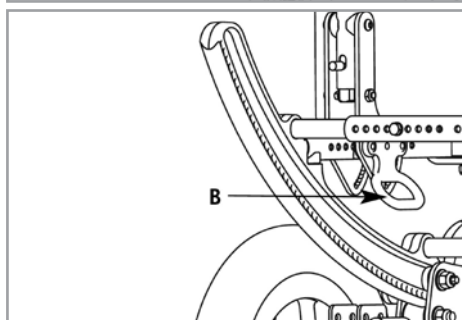
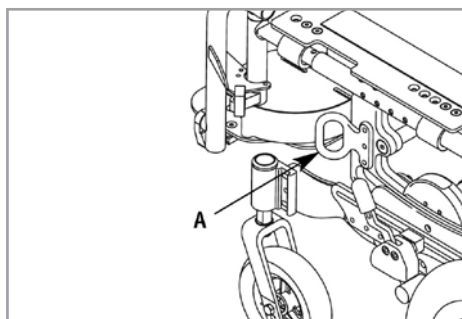
1. If possible and feasible, the rider should transfer to the Original Equipment Manufacturer vehicle seat and use the vehicle restraint.
2. If the wheelchair is used for transport, it must be used in a forward-facing seating position.
3. The rider must not weigh more than 150 lbs.
4. The figure to the right shows the locations of the wheelchair securement points, front (A) and back (B).
5. Use only with Wheelchair Tie down and Occupant Restraint Systems (WTORS) that have been installed in accordance with the manufacturer's instructions and SAE J2249.
6. Attach WTORS to securement points in accordance with the manufacturer's instructions and SAE J2249.
7. Attach occupant restraints in accordance with the manufacturer's instructions and SAE J2249.

USER WEIGHT LESS THAN 22 kg (48 lb 8.0 oz)

When the user being transported is a child, less than 22Kg mass and the vehicle involved has less than eight (8) seated passengers it is recommended that they be transferred to an UNCE Regulation 44 compliant child restraint system (CRS). This type of restraint system provides a more effective occupant restraint system than the conventional 3 point occupant restraint system and some CRS systems also include additional postural supports to assist in maintaining the position of the child when seated.

Parents or care providers may consider the option, in some circumstances, for their child is to remain in their wheelchair whilst in transport due to the level of posture control and comfort provided by the set up in the wheelchair.

We would recommend in such circumstances that a risk assessment be carried out by your healthcare professional and relevant competent persons.

ANTI-TIP TUBES (OPTIONAL)

Anti-tip tubes can help keep your chair from tipping over backward in most normal conditions.

1. Sunrise Recommends Use Of Anti-Tip Tubes:
 - a. UNLESS you are a skilled rider of this chair and are sure you are not at risk to tip over.
 - b. Each time you modify or adjust your chair. The change may make it easier to tip backward. Use anti-tip tubes until you adapt to the change, and are sure you are not at risk to tip over.
2. When locked in place (in the "down" position) anti-tip tubes should be BETWEEN 1 1/2 to 2 inches off the ground.
 - a. If set too HIGH, they may not prevent a tip-over.
 - b. If set too LOW, they may "hang up" on obstacles you can expect in normal use. If this occurs, you may fall or your chair may tip over.
3. Keep Anti-Tip Tubes Locked In Place UNLESS:
 - a. You have an attendant, or
 - b. You have to climb or descend a curb, or overcome an obstacle, and can safely do so without them. At these times, make sure anti-tip tubes are up, out of the way.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

ARMRESTS

Armrests will not bear the weight of this chair.

1. NEVER lift this chair by its armrests. They may come loose or break.
2. Lift this chair only by non-detachable parts of the main frame.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

STROLLER HANDLE

1. Always keep hands away from the locking mechanism located at the bottom of the back canes when folding the back down or pulling it back up.
2. Always keep fingers away from the interior of the hinge when readjusting the adjustable stroller handle.

FASTENERS

Many of the screws, bolts and nuts on this chair are special high-strength fasteners. Use of improper fasteners may cause your chair to fail.

1. ONLY use fasteners provided by an authorized supplier (or ones of the same type and strength, as indicated by the markings on the heads).
2. Over- or under-tightened fasteners may fail or cause damage to chair parts.
3. If bolts or screws become loose, tighten them as soon as you can.
4. **CHOKING HAZARD** – This mobility aid uses small parts which under certain circumstances may present a choking hazard to young children.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

Warnings: Components and Options **WARNINGS!****FOOTRESTS**

Footrests will not bear the weight of this chair

1. At the lowest point, footrests should be **AT LEAST 2 1/2 INCHES** off the ground. If set too **LOW**, they may “hang up” on obstacles you can expect to find in normal use. This may cause the chair to stop suddenly and tip forward.
2. To Avoid A Trip Or Fall When You Transfer:
 - a. Make sure your feet do not “hang up” or get caught in the space between the footrests.
 - b. Avoid putting weight on the footrests, as the chair may tip forward.
3. **NEVER** lift this chair by the footrests. Footrests detach and will not bear the weight of this chair. Lift this chair only by non-detachable parts of the main frame.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

PNEUMATIC Tyres

Proper inflation extends the life of your Tyres and makes your chair easier to use.

1. Do not use this chair if any of the Tyres is under- or over-inflated. Check weekly for proper inflation level, as listed on the Tyre sidewall.
2. Low pressure in a rear Tyre may cause the wheel lock on that side to slip and allow the wheel to turn when you do not expect it.
3. Low pressure in any of the Tyres may cause the chair to veer to one side and result in a loss of control.
4. Over-inflated Tyres may burst.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

POSITIONING BELTS (OPTIONAL)

Use positioning belts **ONLY** to help support the rider’s posture. Improper use of these belts may cause severe injury to or death of the rider.

1. Make sure the rider can not slide down in the wheelchair seat. If this occurs, the rider may suffer chest compression or suffocate due to pressure from the belts.
2. The belts must be snug, but must not be so tight that they interfere with breathing. You should be able to slide your open hand, flat, between the belt and the rider.
3. A pelvic wedge or a similar device can help keep the rider from sliding down in the seat. Consult with the rider’s doctor, nurse or therapist to find out if the rider needs such a device.
4. Use positioning belts only with a rider who can cooperate. Make sure the rider can easily remove the belts in an emergency.
5. **NEVER** Use Positioning Belts:
 - a. As a patient restraint. A restraint requires a doctor’s order.
 - b. On a rider who is comatose or agitated.
 - c. As a motor vehicle restraint. In an accident or sudden stop the rider may be thrown from the chair. Wheelchair seat belts will not prevent this, and further injury may result from the belts or straps.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

Warnings: Components and Options **WARNINGS!****POWER DRIVE**

- Do not install a power drive on any Zippie wheelchair. If you do:
1. It will affect the center of balance of your chair and may cause a fall or tip-over.
 2. It will alter the frame and void the warranty.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

PUSH HANDLES

When you have a attendant, make sure that this chair has push handles.

1. Push handles provide secure points for an attendant to hold the rear of this chair, to prevent a fall or tip-over. Make sure to use push handles when you have an attendant.
2. Check to make sure push handle grips will not rotate or slip off.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

QUICK-RELEASE AXLES

1. Do not use this chair **UNLESS** you are sure that both quick-release rear axles are locked. An unlocked axle may come off during use and cause a fall.
2. An axle is not locked until the quick-release button pops out fully. An unlocked axle may come off during use, resulting in a fall, tip-over or loss of control and cause severe injury to the rider or others.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

REAR AXLES

A change in set-up of the rear wheels will affect the center of balance of your chair.

1. The farther you move the rear axles **FORWARD**, the more likely it is that your chair will tip over backward.
2. Consult your doctor, nurse or therapist to find the best rear axle set-up for your chair. Do not change the set-up **UNLESS** you are sure you are not at risk to tip over.
3. Adjust the rear wheel locks after you make any change to the rear axles.
 - a. If you fail to do so, the locks may not work.
 - b. Make sure lock arms embed in Tyres at least 1/8 inch when locked.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

 **WARNINGS!****REAR WHEEL LOCKS**

Rear wheel locks are NOT designed to slow or stop a moving wheelchair. Use them only to keep the rear wheels from rolling when your chair is at a complete stop.

1. NEVER use rear wheel locks to try to slow or stop your chair when it is moving. Doing so may cause you to veer out of control.
2. To keep the rear wheels from rolling, always set both rear wheel locks when you transfer to or from your chair.
3. Low pressure in a rear Tyre may cause the wheel lock on that side to slip and may allow the wheel to turn when you do not expect it.
4. Make sure lock arms embed in Tyres at least 1/8 inch when locked. If you fail to do so, the locks may not work.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

MODIFIED SEAT SYSTEMS

Use of a seat system not approved by Sunrise may alter the center of balance of this chair. This may cause the chair to tip over.

1. Do not change the seat system of your chair UNLESS you consult your authorized supplier first.
2. Use of a non-validated system (wheelchair base and seating), as a seat in a motor vehicle could result in increased risk of severe injury or death in a vehicular mishap.
 - Only a complete system (wheelchair base and seating) can be validated for use as a seat in a motor vehicle.
 - The seating provided may or may not be appropriate for use as a seat in a motor vehicle in combination with this wheelchair base.
 - Before using this system as a seat in a motor vehicle, you should verify that the seating manufacturer has validated the seating in combination with this wheelchair base.
 - Before using this system as a seat in a motor vehicle, you should read and comply with all instructions for use provided by both Sunrise Medical and the seating manufacturer as well as the instructions for use of any other component or accessory manufacturer.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

 **WARNINGS!****TRANSIT OPTION**

1. Use only Wheelchair Tiedown and Occupant Restraint Systems (WTORS) which meet the requirements of SAE J2249 Recommended Practice - Wheelchair Tiedown and Occupant Restraint Systems For Use in Motor Vehicles. Do not use WTORS designed to rely on the wheelchair structure to transfer occupant restraint loads to the vehicle.
2. The wheelchair has been dynamically tested in a forward-facing mode for a 30 mph frontal impact test. The wheelchair must be forward-facing during transport.
3. In order to reduce the potential of injury to vehicle occupants, wheelchair-mounted accessories, such as trays and respiratory equipment, should be removed and secured separately.
4. Postural supports and positioning devices should not be relied on for occupant restraint.
5. Do not alter or substitute wheelchair frame parts, components or seating.
6. Sudden stops or impacts can structurally damage your chair. Chairs involved in such incidents should be replaced.
7. When the user being transported is a child, less than 22Kg mass and the vehicle involved has less than eight (8) seated passengers it is recommended that they be transferred to an UNCE Regulation 44 compliant child restraint system (CRS). This type of restraint system provides a more effective occupant restraint system than the conventional 3 point occupant restraint system and some CRS systems also include additional postural supports to assist in maintaining the position of the child when seated. Parents or care providers may consider the option, in some circumstances, for their child is to remain in their wheelchair whilst in transport due to the level of posture control and comfort provided by the set up in the wheelchair. We would recommend in such circumstances that a risk assessment be carried out by your healthcare professional and relevant competent persons.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

Set-up and Adjustment

NOTE: Setting the wheelchair on a table or on a flat surface, such as a workbench, helps make these procedures easier.

NOTE ON TORQUE SETTING: A torque setting is the optimum tightening which should be made on a particular fastener. A torque wrench which measures in./lbs. should be used when tightening fasteners on Zippie wheelchairs. Unless otherwise noted, use a torque setting of 60 in./lbs. When setting up this wheelchair.

SWING-AWAY HANGERS AND FOOTRESTS 60°, 70°, 70°V, 80°, OR 90°

1. Installation

- Place swing-away pivot saddle (A) into the receiver (B) on front frame tube with the footrest facing outward from frame.
- Rotate the footrest inward until it locks into place on locking bolt (C).

2. Removal

- To remove footrest, push release latch (D) toward the frame.
- Rotate footrest outward and lift.

3. Height Adjustment with Extension Tube

- Remove both bolts (E) in the hanger frame tube.

- Reposition footrest height.

- Re-tighten all bolts (E).

4. Height Adjustment, Front Mount

- Loosen and remove the retaining bolt (F) in the frame tube.
- Position footplate upward or downward in predrilled holes on hanger tube.

- Reinsert bolt (F) and tighten.

HEAVY-DUTY, LIFT-OFF FOOTRESTS (OPTIONAL): 70°, 80°, 90°

1. Installation:

Slide pivot saddle (A) into receiver (B) on front frame tube until it locks into place.

2. Removal:

To remove footrest, push release latch (C) while lifting up footrest from receiver.

3. Height-Adjustment, Front Mount

- Loosen and remove the retaining bolt (D) in the footrest mounting bracket.

- Position footplate upward or downward in predrilled holes on hanger tube.

- Reinsert bolt and tighten.

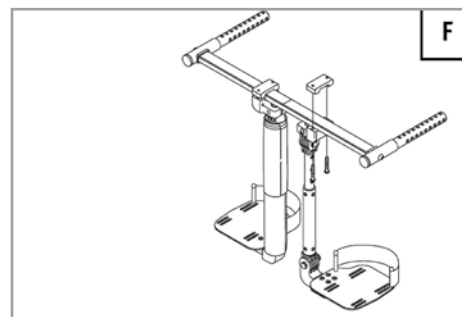
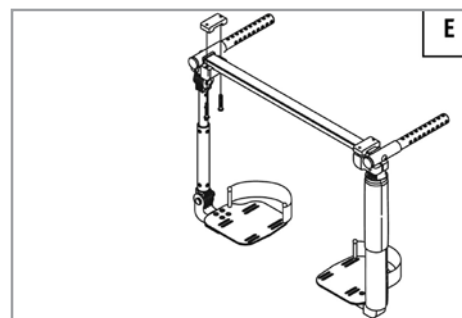
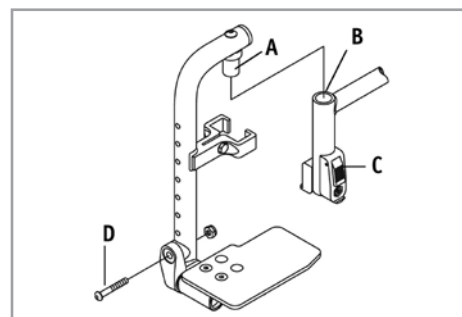
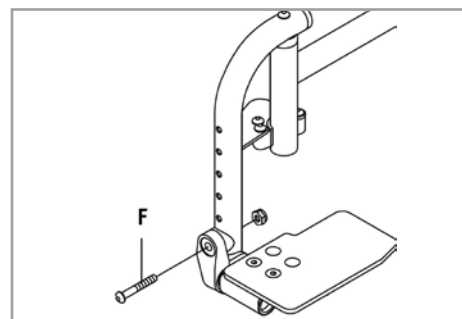
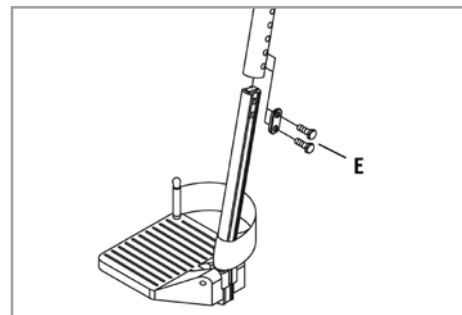
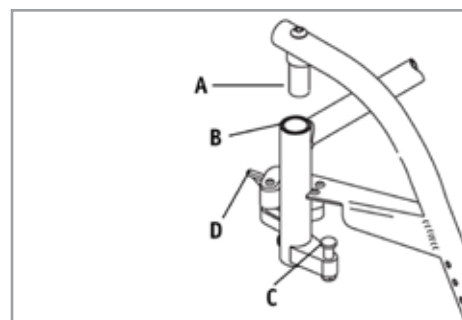
CONTRACTURE FOOTREST (OPTIONAL)

1. Lateral Mount:

Can be mounted laterally as shown in illustration (E).

2. Center Mount:

Can be mounted in the center as shown in illustration (F).

Set-up and Adjustment

HEIGHT ADJUSTMENT

- a. Loosen and remove the retaining bolt (G) in the footrest mounting bracket.
- b. Slide the lower tube upward or downward to the desired position until predrilled holes on hanger tubes are aligned.
- c. Reinsert bolt and tighten.
4. Footrest Angle Adjustment
 - a. Loosen the retaining bolt (H) in the adjustment locking joint.
 - b. Position footrest forward or rearward in desired angle.
 - c. Retighten the bolt.
5. Footplate Angle Adjustment
 - a. Loosen the retaining bolt (I) in the footplate locking joint.
 - b. Position footplate upward or downward in desired angle.
 - c. Retighten the bolt.

ARTICULATING LEGREST (OPTIONAL)

1. Installation or Removal

To install or remove Articulating Legrest (ALR) see instructions for Swing-away Footrest installation.

2. Elevation Adjustment

- a. To raise legrest, lift to desired position. Legrest will automatically lock in place.
- b. To lower legrest, while seated in chair, press release lever (A) down and lower legrest to desired position. Legrest will automatically lock in place.

3. Height Adjustment of the Calf Pad

- a. Rotate the tube clamp (B) towards the outside.
- b. Slide it up or down.
- c. Rotate it back in the desired indented position.

4. Height Adjustment with Extension Tube

- a. Remove the bolt (C) in the hanger frame tube.
- b. Reposition footrest height.
- c. Align the tube clamp with the selected bolt.
- d. Re-tighten the bolt (C).

ELEVATING LEGREST (OPTIONAL)

1. Installation or Removal

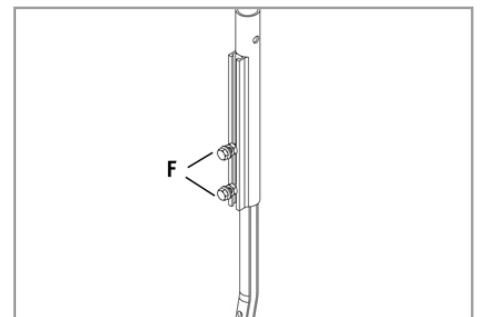
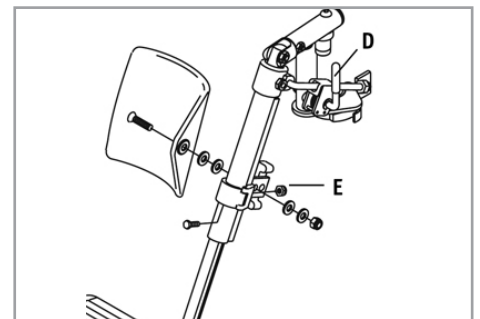
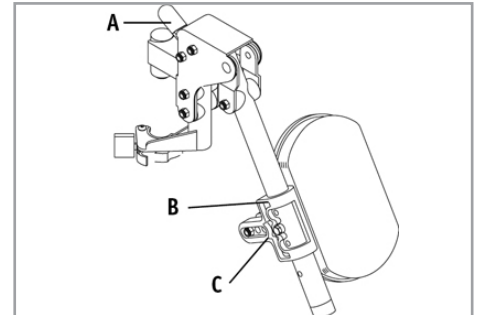
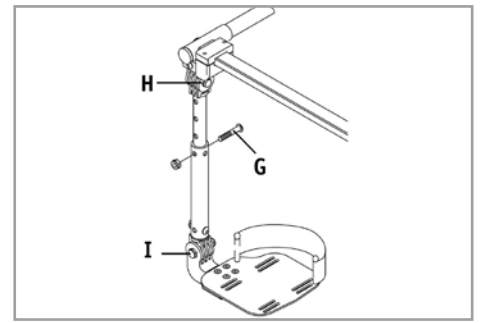
To install or remove Elevating Legrest (ELR) see instructions for Swing-away Footrest installation.

2. Elevation Adjustment

- a. To raise legrest, lift to desired position. Legrest will automatically lock in place.
- b. To lower legrest, while seated in chair, press release lever (D) down and lower legrest to desired position. Legrest will automatically lock in place.

3. Height Adjustment of the Calf Pad.

- a. Loosen the retaining nut (E) located at the back of the calf pad bracket assembly.
- b. Slide the calf pad bracket assembly up or down.
- c. Tighten it in the desired position.
4. Height Adjustment with Extension Tube
 - a. Loosen both bolts (F) in the hanger frame tube.
 - b. Reposition footrest height.
 - c. Re-tighten all bolts (F).

Set-up and Adjustment

CASTER/FORKS ASSEMBLY

1. Installation

- Remove dust cover (A) from caster housing. Pry dust cover off using a back and forth motion.
- Caster assembly holds one nut (B) and one washer (immediately below the nut). Remove nut and washer.
- Insert caster stem (C) into caster housing (D) and place washer on stem.
- Replace nut (B) on caster stem and tighten until all play is removed; then loosen approximately 1/8 turn, yet still allowing for free rotation.
- Replace dust cover to caster housing.

2. Caster Angle Adjustment

For optimum performance, the caster housing should always be at a 90° angle to the floor (perpendicular to the ground). This adjustment is made with an eccentric (offset) attachment bolt mechanism.

- To change the angle, place the chair on a flat surface (such as a table).
- Remove the bolt nuts (G) from the back side of the caster plate. The caster plate is slotted at the top and bottom for adjustment.
- Place a large right triangle against the table surface and the front surface of the caster bearing housing to align the caster stem rotational axis perpendicular to the floor surface.
- Rotate the eccentric washers (E) independently until they fall into position.
- When the alignment is correct, retighten the two retaining bolts.
- Repeat the process for the other caster.

NOTE: Use a torque setting of 100 in.-lbs. when setting-up caster plate.

3. Height Adjustment

- Loosen and remove bolt (F).
- Set axle at desired height using pre-set holes in the fork.
- Replace and tighten bolt.
- Repeat on other side.

NOTE: The frame must be kept horizontal when height is adjusted. Refer to matrix in the ordering guide for proper setting.

DUAL-POST ARMRESTS

1. Installation or Removal

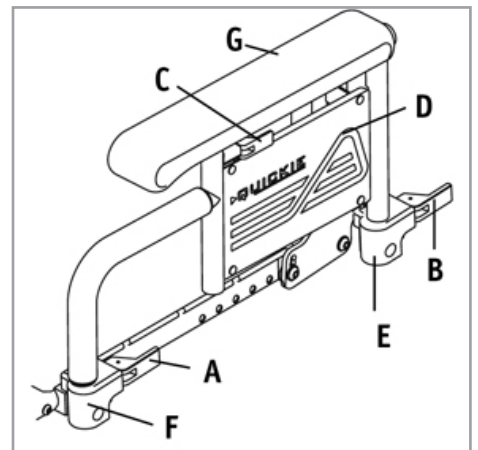
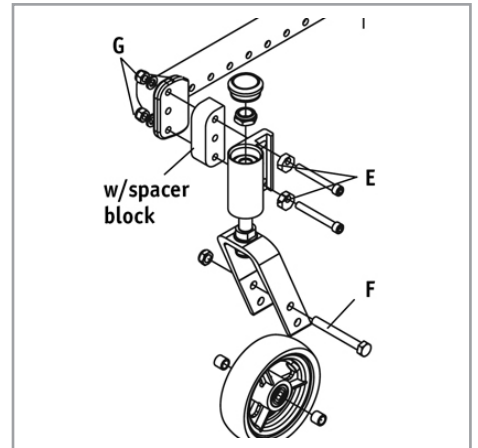
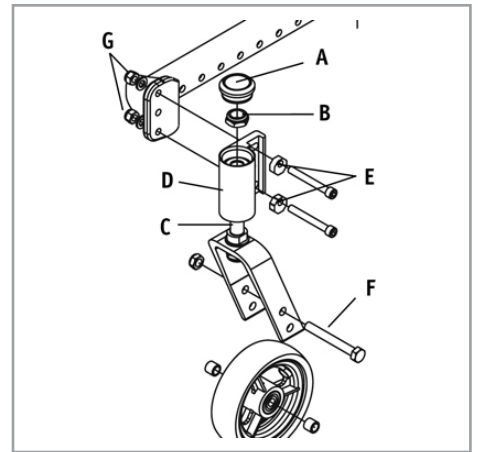
- The assembly is held securely in place when the securing levers (A, B) are facing outside.
- Pull securing levers forward to release.
- Insert or remove armrest.
- Return securing levers to locked position.

2. Height-Adjustment (optional)

- Release the upper securing lever (C).
- Set armrest at desired height.
- Return securing lever to locked position.
- Move armrest up or down to allow armrest to snap into place.

3. Flip Back (optional)

- Release the front securing lever only (A).
- Flip the armrest back, it remains attached to the rear receiver.
- Move armrest down into place.
- Return securing lever to locked position.



Dual Post Armrest

- A. Front release latch
- B. Rear release latch
- C. Upper securing lever
- D. Armrest panel
- E. Rear receiver
- F. Front receiver
- G. Armrest pad

Set-up and Adjustment

SINGLE-POST HEIGHT ADJUSTABLE ARMRESTS

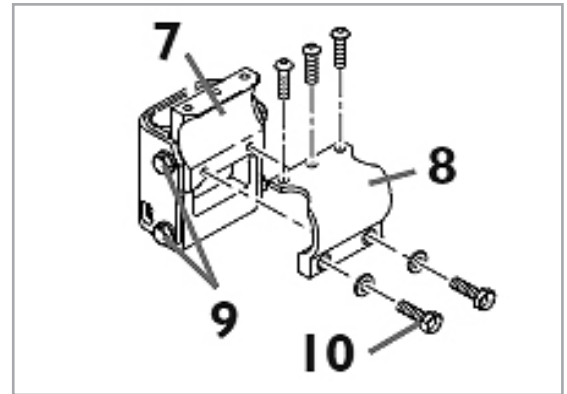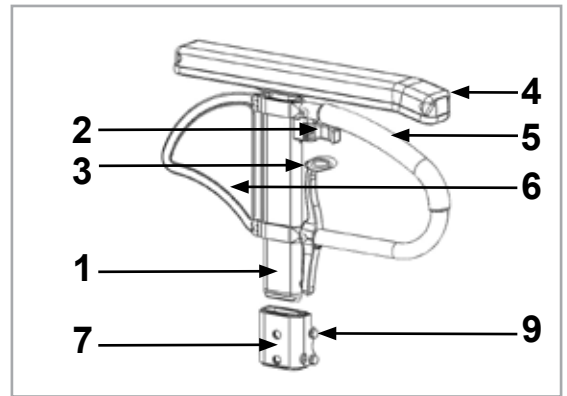1. Fitting
 - a. Push the outer armrest rails down into the receiver which is mounted on the wheelchair frame.
 - b. The armrest will automatically lock into place.
2. Height adjustment
 - a. Turn the release lever for height adjustment (2) to the second stop point.
 - b. Push the armpad up or down to reach the desired height.
 - c. Set the lever back to the interlocked position against the armrest rails.
 - d. Push the armpad (4) until the rails click into place securely.
3. Remove the armrest
 - a. Operate the lever 3 and lift the whole arm.
4. Insert armrest
 - a. Put the armrest back in the receiver, until the arm clicks into place.

MOUNTING COMPONENTS FOR THE ARMREST RECEIVER

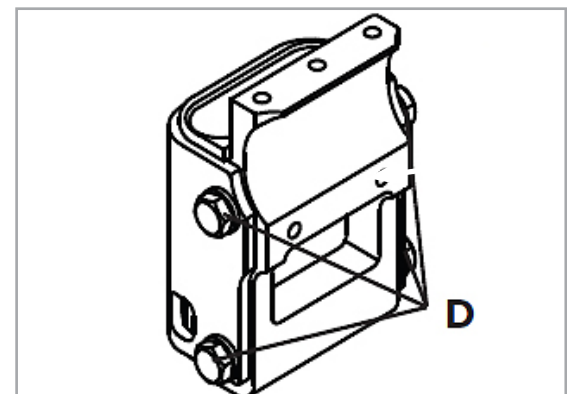
Adjusting the armrest receiver

To tighten or loosen the fit of the outer armrest rails in the receiver:


1. Loosen the four set screws for the receiver (D) on the sides of the receiver.
2. Leave the armrest in the receiver (E) and push the receiver together, until the desired adjustment is achieved.
3. Tighten the four screws (D). (144 in-lbs, 16.3 Nm)

Position adjustment

1. Loosen the two clamp screws (10) until the clamp is loose.
2. Push the armrest receiver to the desired position.
3. Tighten.
(Fig.38, 39, 40 and 41).

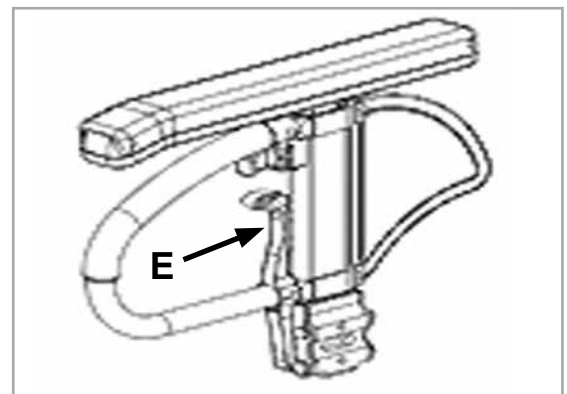1. Outer armrest rails
2. Height adjustment of the locking lever
3. Locking lever
4. Armrest pad
5. Transfer bar
6. Side-guard
7. Receiver
8. Adjustment clamp
9. Parts to adjust the receiver
10. Clamp screws

Set-up and Adjustment

ANTI-TIP TUBES- REAR

Anti-tip tubes are recommended for all wheelchairs.

NOTE: Use a torque setting of 96 in.-lbs. when setting-up anti-tip tubes.

1. Inserting Anti-Tip Tubes Into Receivers

- Press the rear anti-tip release pin (A) on the anti-tip tube so that both release pins are drawn inside.
- Insert the anti-tip tube into the receiver (B).
- Turn the anti-tip tube down until release pin protrudes through the receiver mounting hole.
- Insert second anti-tip tube the same way.

2. Turning Anti-Tip Tubes Up

Turn anti-tip tubes up when being pushed by attendant, overcoming obstacles or climbing curbs.

- Press the rear anti-tip tube release pin (A).
- Hold pin in and turn anti-tip tube up (C).
- Release pin.
- Repeat with second anti-tip tube.
- Remember to return anti-tip tubes to down position after completing maneuver.

3. Adjusting Anti-Tip Tube Wheel

The anti-tip tube wheels may have to be raised or lowered to achieve proper ground clearance (1-1/2" to 2").

- Press the anti-tip wheel release pin (D) so that the release pin is drawn inside.
- Raise or lower to one of the three or four pre-drilled holes.
- Release pin (D).
- Adjust the second anti-tip tube wheel the same way. Both wheels should be at exactly the same height.

4. Height adjustment of the receivers

- Remove anti-tip tubes (E) from anti-tip receivers (B).
- Remove bolts (F) from anti-tip tube receiver.
- Reposition bracket to desired height. The receiver can be positioned up or down as needed.
- Reinsert bolts (F) and tighten.
- Repeat with second anti-tip tube receiver. Both brackets should be at exactly the same height.
- Replace anti-tip tubes.

ANTI-TIP TUBES- FRONT

NOTE: Use a torque setting of 100 in.-lbs. when setting-up anti-tip tubes.

1. Inserting Anti-Tip Tubes Into Receivers

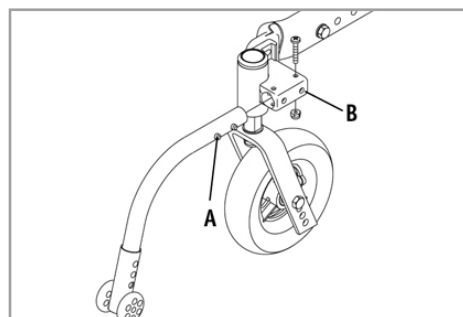
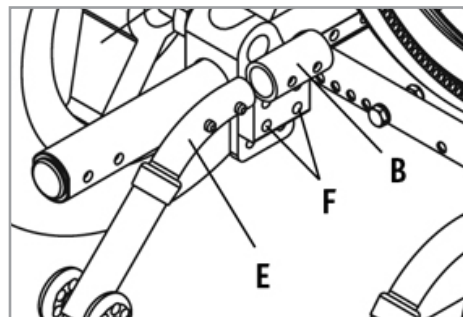
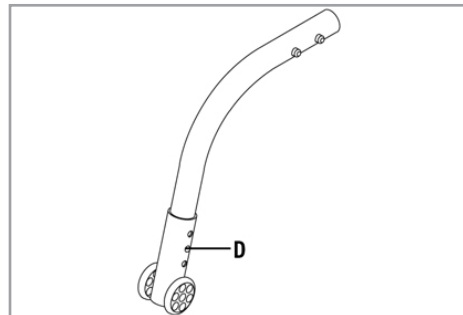
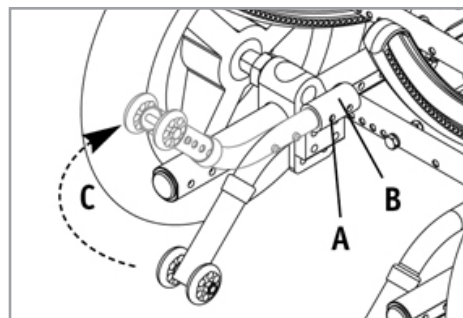
- Press the front anti-tip release pin (A) on the anti-tip tube so that both release pins are drawn inside.
- Insert into the anti-tip tube receiver (B).
- Turn the anti-tip tube down until release pin is through the receiver mounting hole.
- Insert second anti-tip tube the same way.

2. Turning Anti-Tip Tubes Up

See Turning Anti-tip Tubes Up section J-2.

3. Adjusting Anti-Tip Tube Wheel

See adjusting Anti-Tip Tube Wheel section J-3.

Set-up and Adjustment

TILT-IN-SPACE MECHANISM

The tilt-in-space mechanism is installed by Sunrise. Two cable/trigger mechanisms positively lock the chair in place from -5° to 60° depending on the set-up. The Zippie IRIS has the capability to provide from 0° to 5° of forward tilt.

WARNING!

Do not attempt to make this adjustment.
This adjustment must be done by an authorized supplier.

1. Tilt Operation

- Depressing both triggers (C) at the same time disengages the locking mechanism and allows the seat and back to rotate in space. Releasing the triggers positively locks the rockers in place.
- The approximate degree of tilt the seat is at can be seen on the tilt indicator (D).
- To lock out the tilting function, insert the locking pin (E) into the trigger holder.

2. Tilt-stops Adjustment

Adjustments can be made to limit the amount of tilt (rotation) that the chair provides by using the tilt-stops (A).

- Loosen and remove the retaining screw (B).
- Locate the tilt-stops to the desired limit either anterior or posterior.
- Each half of tilt stop assembly should plug into directly opposing holes on the rocker (C), then secure in place by tightening the screw (B).

3. Cable Adjustment

NOTE: Do not use lubricants on the components or assembly.

- Loosen and remove nuts (D) and remove bracket assembly (E).
- Loosen the jam nut (F).
- Rotate the cable adjuster (G) until slack is removed from the cable. Make sure the plunger pin can fully extend and retract after adjustment.
- Tighten jam nut (F).
- Reinstall the bracket assembly (E).
- Install and tighten nuts (D).

HEIGHT-ADJUSTABLE BACK WITH ADJUSTABLE STROLLER HANDLE, (OPTIONAL).

1. Angle Adjustment

- Loosen rear bolt (F).
- Loosen and remove front bolt (G).
- Set backrest at desired angle using pre-set holes.
- Replace and tighten the front bolt (G).
- Tighten rear bolt (F).
- Repeat on other side.

IMPORTANT NOTE: The adjustment on each side of the chair should be exactly equal.

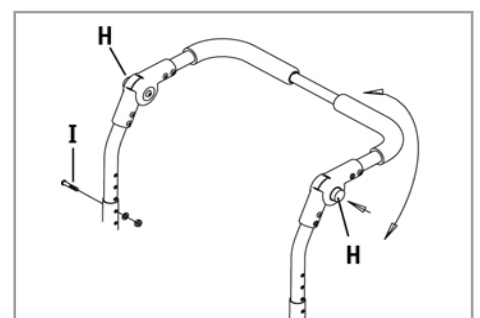
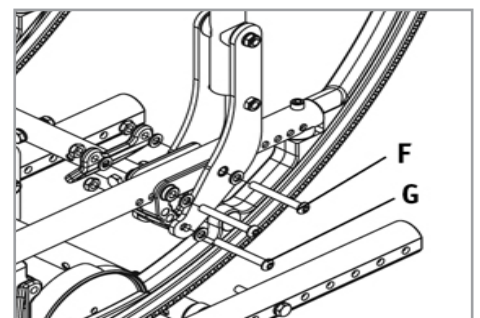
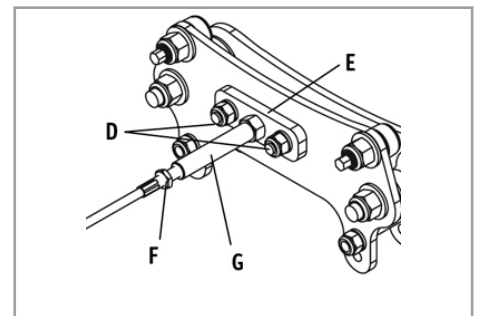
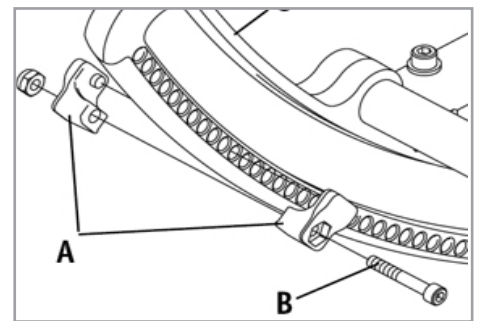
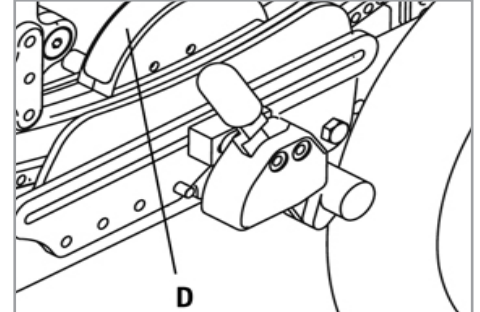
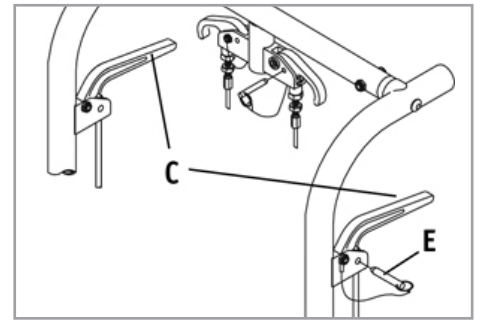
NOTE: Use a torque setting of 96 in.-lbs. when setting-up the angle adjustment of the stroller handle.

2. Angle Adjustment of the Stroller Handle

Depress both black push buttons (H) at the same time. This disengages the stroller handle and allows it to rotate freely. Releasing the push buttons positively locks the adjustable stroller handle in the desired position.

3. Height Adjustment

- Remove both bolts (I) from the backrest tube (one on each side).
- Reposition backrest height.
- Re-tighten both bolts (I) in the desired location.

Set-up and Adjustment

FLIP-DOWN BACK FUNCTION

- To fold the back down, pull-up the back release pins (A) and push the backrest forward.
- To lock it back in position, lift the backrest firmly until it locks in its original position.

STROLLER HANDLE EXTENSION (OPTIONAL)

Installation

- Press button (B) on the stroller handle so that release pins are drawn inside the tube.
- Insert the tube into the back tube receiver (C) until the release pin protrudes through the receiver mounting hole.

RECLINING BACK

The dual trigger reclining mechanism is installed by Sunrise.

1. To Recline or Raise Backrest

- Depress both triggers (D) at the same time. Releasing the triggers positively locks the backrest in place within its 45° range.
- To lock out the adjustment system, insert the locking pin (E) into the trigger holder. Always lock the dual trigger reclining mechanism securely in place after positioning the backrest.

FRAME DEPTH

1. Back adjustment

- Loosen and remove the front bolt (F) located in front of back mounting plate.
- Loosen and remove the rear bolt (H) from the back mounting plate.
- Repeat both steps on the other side.
- Slide the backrest along the seat rail to the desired position.
- Replace and tighten the front bolt (F) and the rear bolt (H).
- Repeat on the other side.

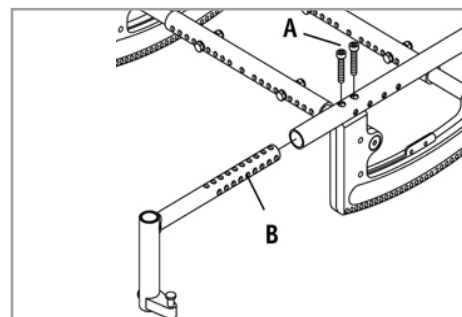
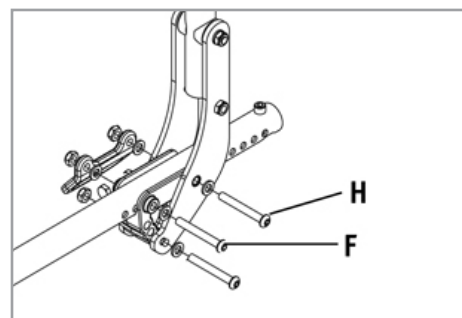
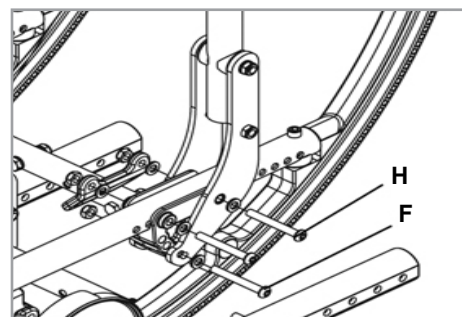
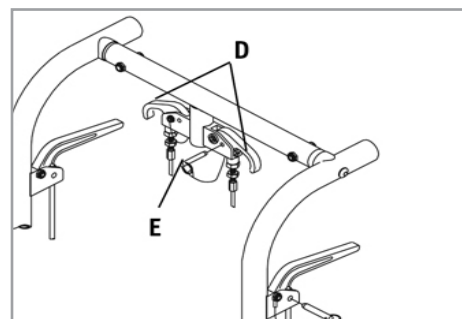
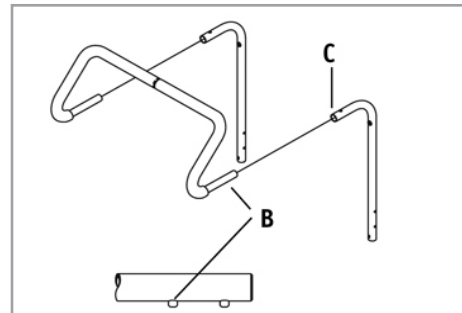
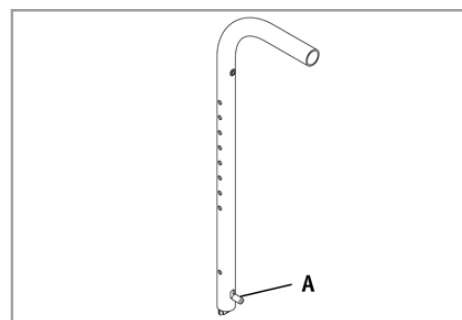
IMPORTANT NOTE: The adjustment on each side of the chair should be exactly equal.

2. Hanger Receiver Adjustment

- Loosen and remove the socket head screws (A) from the seat rail.
- Slide the hanger receiver (B) in the seat rail to the desired position.
- Replace and tighten the socket head screws.
- Repeat on the other side.

IMPORTANT NOTE: These adjustments may also require re-positioning the carriage over the base frame, refer to matrix in the ordering guide for recommended location.

NOTE: Depending on the original position of the hanger receiver and the amount of adjustment required, a new set of seat tubes may be required. Holes in the hanger receiver should not be visible past the front of the seat tube.

Set-up and Adjustment

FRAME WIDTH

The frame of the IRIS is built to accommodate growth from 10" to 18" wide using the three strut tube assemblies. Moving one hole position on each side of the strut provides a 1" frame width change.

1. Width Adjustment

- Remove the bolts (C) from one side of the strut tubes (D) for all strut tubes.
- Slide the adjustable extension (E) to the desired width according to the matrix in the ordering guide.
- Replace and tighten the bolts in all strut tubes.
- Repeat on the other side.

IMPORTANT NOTE: The adjustment on each side of the chair should be exactly equal and the alignment of the holes must remain horizontal.

NOTE– Use a torque setting of 96 in.-lbs. when setting up frame width.

SEAT PAN

The seat pan of the IRIS is designed to accommodate depth adjustment.

2. Depth adjustment

- Loosen and remove all bolts (F).
- Adjust the seat pan to the desired depth.
- Replace and tighten all bolts.

CUSHION INSTALLATION

- Place cushion on seat pan with hook material side down. The beveled edge of the cushion should be in front.
- Press firmly into place.

REAR AXLE

1. To Mount and Remove Rear Wheels (optional)

Do not use this chair UNLESS you are sure both quick-release axles are locked. An unlocked axle may come off during use and cause a fall.

NOTE: Axle is not locked until the quick-release button (A) pops out fully.

2. To Install Wheel

- Depress quick-release button (A) fully. This will release tension on ball bearings at other end.
- Insert axle through hub of rear wheel (B).
- Keep pressure on button (A) as you slide axle (C) into axle sleeve (D).
- Release button to lock axle in sleeve. Adjust axle if it does not lock.
- Repeat steps on other side.

3. To Remove Wheel

- Depress quick-release button (A) fully.
- Remove wheel by sliding axle (C) out of axle sleeve (D).
- Repeat steps on other side.

4. Rear Wheel Axle Adjustment

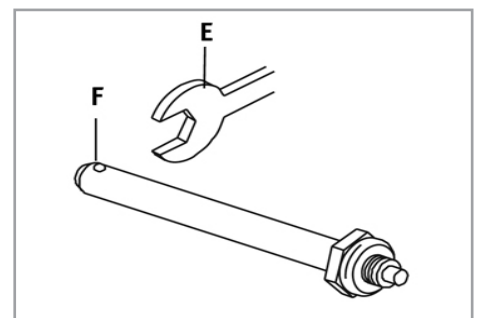
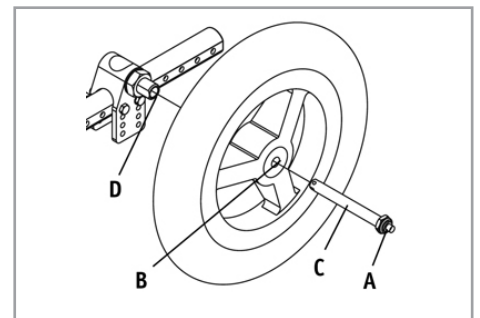
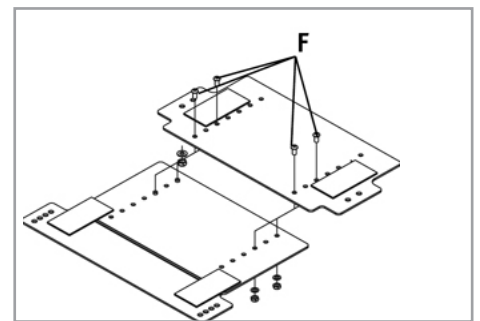
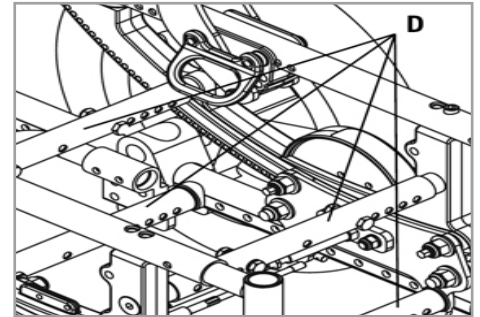
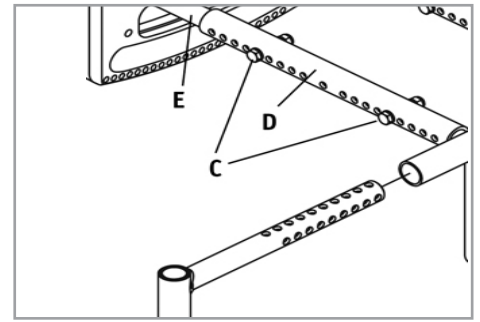
Tight axle sleeves should be maintained for proper performance of the wheelchair.

- To adjust the axle you will need a 3/4" wrench to turn the outside axle nuts.
- You will also need a 1/2" wrench (E) to lodge the ball bearings (F), on the opposite end of the axle, and prevent the axle from turning.
- Turn the outside axle nut counter-clockwise to tighten.
- There should only be zero to ten thousandths of an inch (.010) of play.

5. Weight Distribution

Weight distribution between the front and rear wheels is primarily adjusted by moving the rear axles forward or rearward on the frame.

Moving the axles REARWARD provides better forward and rearward tipping stability.

Set-up and Adjustment

Moving the axles FORWARD shifts more of the rider's weight onto the REAR wheels, providing the following benefits: the front end of the wheelchair becomes lighter, thus reducing the turning effort, and with a shorter wheelbase the turning radius is reduced.

AXLE ADJUSTMENT

- Remove both wheels from the chair.
- Remove the bolts (A) from the axle plate (B).
- Reposition the axle plate in desired position.
- Insert and tighten bolts (A).
- Repeat on the other side.

IMPORTANT NOTE: The adjustment on each side of the chair should be exactly equal.

IMPORTANT NOTE: Do not attempt to make this adjustment. This adjustment must be done by an authorized supplier only.

NOTE: Use a torque setting of 96 in.-lbs. when setting-up the axle plate.

AXLE HEIGHT ADJUSTMENT

- Loosen and remove bolts (A).
- Set axle at desired height using pre-set holes in the axle plate (B).
- Replace and tighten bolts.
- Repeat on other side.

IMPORTANT NOTE: The adjustment on each side of the chair should be exactly equal.

IMPORTANT NOTE: Do not attempt to make this adjustment. This adjustment must be done by an authorized supplier only.

NOTE: Use a torque setting of 96 in.-lbs. when setting-up the axle plate.

NOTE: The frame must be kept horizontal when height adjustments are made. Refer to matrix in the ordering guide for proper setting.

CARRIAGE

The carriage of the Zippie IRIS can be adjusted to preset the range of tilt from -5° to 50° from 0° to 55° or from 5° to 60° .

IMPORTANT NOTE: Do not attempt to make this adjustment. This adjustment must be done by an authorized supplier only.

1. Adjustment

- Loosen and remove front bolt (A) and rear bolt (B) from the carriage on each side.
- Set the carriage at desired range according to range settings below.
- Replace and tighten all bolts.

IMPORTANT NOTE: The adjustment on each side of the chair should be exactly equal.

NOTE: Use a torque setting of 96 in.-lbs. when setting-up carriage.

2. Range and setting

- Figure C: -5° to 50°
- Figure D: 0° to 55°
- Figure E: 5° to 60°

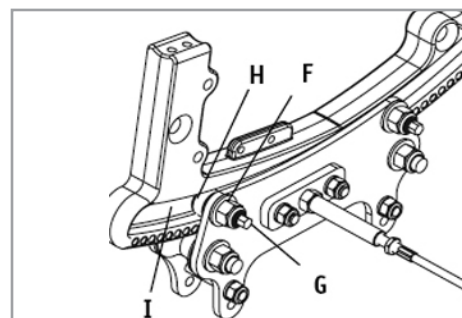
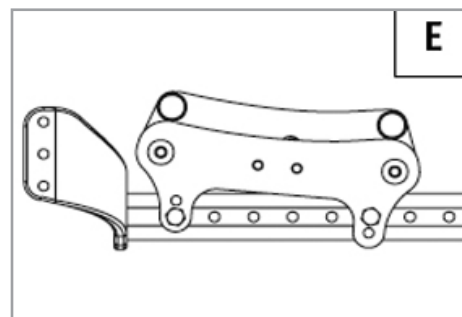
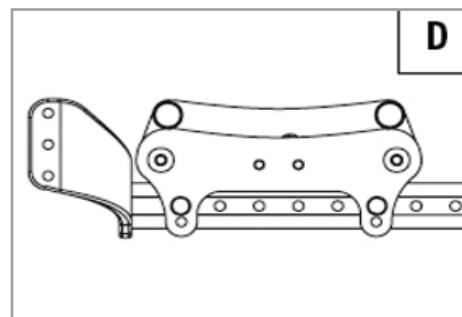
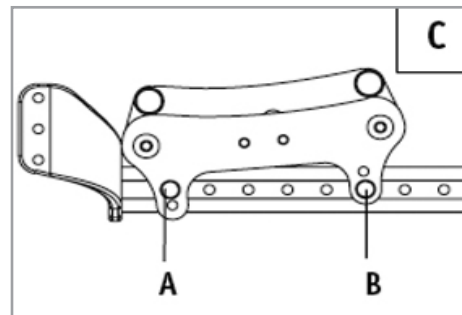
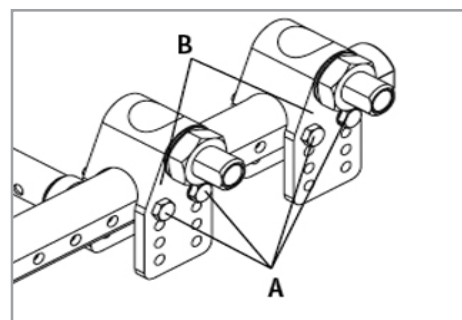
3. Top Roller Adjustment

Tools recommended: (1) 1/4" open-end wrench and (1) 9/16" open-end wrench.

IMPORTANT NOTE: Remove the rider from the chair before performing this maintenance.

NOTE: It is highly recommended to adjust all 4 of the top rollers at the same time.

- Loosen the cam nut (F).
- Use a 1/4" wrench to rotate the eccentric cam bolt (G) clockwise until the top roller (H) just makes contact with the rocker (I).
- While holding the 1/4" wrench, tighten the 9/16" nut (F) to approximately 120 in-lbs of torque to lock the assembly into place. Although the top roller should be touching the rocker, there should be NO CONTACT PRESSURE of the top roller against the rocker when the assembly is in the tightened condition. The top roller should be able to rotate by applying mild finger pressure.

Set-up and Adjustment

TOP ROLLER REPLACEMENT

NOTE: It is highly recommended to replace both top rollers within each carriage assembly at the same time. Do not use lubricants anywhere on the assembly.

- Loosen both plunger pin bracket nuts (J).
- Loosen and remove both bottom roller nuts (K) and both chassis fastener nuts (L).
- Slide the inner carriage plate (M) off the fasteners, towards the center of the chair.
- Loosen and remove both eccentric cam nuts (N).
- Remove old top rollers (O) from the eccentric cam bolts (P) and replace with new top rollers.
- Reinstall the eccentric cam bolts (P) onto the inner carriage plate (M), installing nuts (N) only finger tight.
- Reinstall the carriage plate (M) onto the frame assembly (Q). Replace and tighten the nuts (K, L).
- Ensure that the plunger pin is centered and engaged into a rocker hole and tighten plunger pin bracket nuts (J).
- Adjust the top roller assemblies per number 3: Top Roller Adjustment.
- Repeat on the other side of chair as necessary.

NOTE: Use a torque setting of 120 in.-lbs. when setting up top roller.

BOTTOM ROLLER REPLACEMENT

NOTE: It is highly recommended to replace both bottom rollers within each carriage assembly at the same time. Do not use lubricants anywhere on the assembly.

- Loosen both plunger pin bracket nuts (J).
- Loosen both eccentric cam nuts (N).
- Loosen and remove both bottom roller nuts (K) and both chassis fastener nuts (L).
- Slide the inner carriage plate (M) off the fasteners, towards the center of the chair.
- Remove old bottom rollers (R) from the roller bushings (S) and replace with new bottom rollers.
- Reinstall the carriage plate (M) onto the frame assembly (Q). Replace and tighten the nuts (K, L).
- Ensure that the plunger pin is centered and engaged into a rocker hole and tighten plunger pin bracket nuts (J).
- Adjust the top roller assemblies as per number 3: Top Roller Adjustment.
- Repeat on the other side of chair as necessary.

NOTE: Use a torque setting of 180 in.-lbs. when tightening bottom roller nuts (K).

WHEEL LOCKS

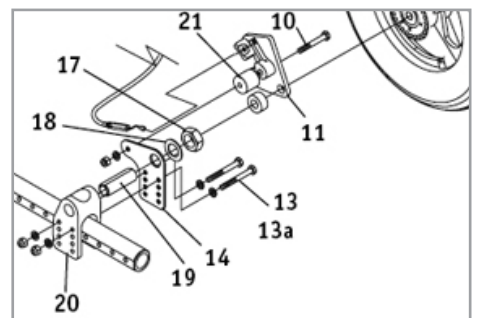
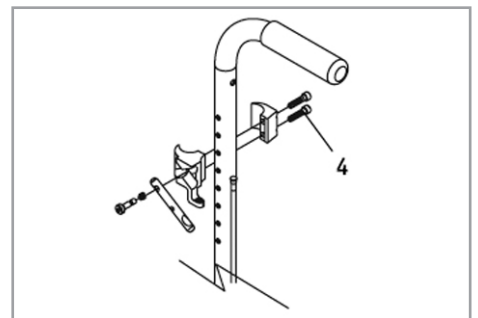
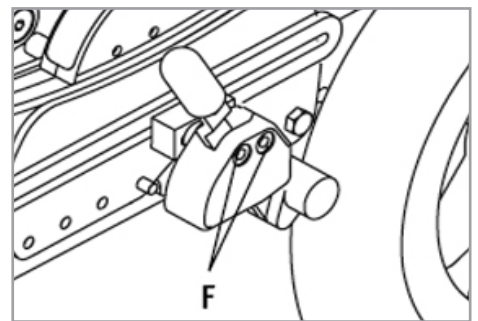
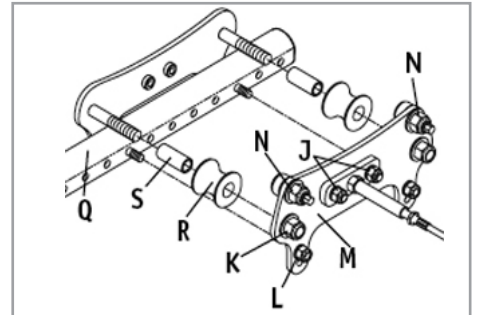
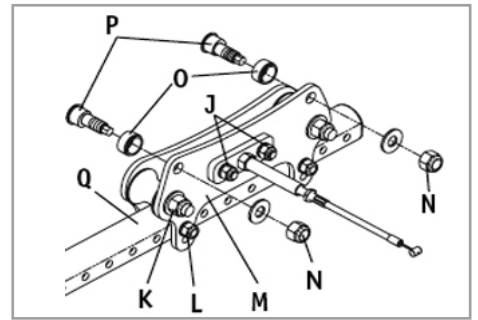
Zippie IRIS wheelchairs are standard with wheel locks. Wheel locks are installed at Sunrise.

Adjustment

- Loosen the screws (F). Do not attempt to remove one screw at a time.
- Slide assembly toward rear wheel until clamp embeds into Tyre to prevent wheel movement, when in locked position.
- Tighten screws.

HUB LOCK INSTALLATION

- Remove wheel and axle from chair.
- Remove 1/4" x 2" long bolts from axle plate (item 20). Note position and orientation of the axle plate.
- Remove 3/4" Nut (item 17) and washer (item 18) from axle sleeve (item 19).
- Place Hub Lock Axle Plate Adapter (item 14) against axle plate and attach axle plate and adapter to frame using bolts supplied (item 13 or 13A). If anti-tips are to be installed, the bolts will be 1/4" x 2-3/4" long. If no anti-tips are used the bolts will be 1/4" x 2-1/4" long. Secure new bolts using washers along with new 1/4" lock nuts as shown.
- Thread axle sleeve further into axle plate such that when the 3/4" nut is tightened, the face of the axle sleeve protrudes out of the 3/4" Nut by approximately 1/32". Tighten 3/4" Nut.
- Discard the large inner diameter spacer and 1/4" x 3" bolt provided with the Lower Hub Lock Assembly (item 11). Place nose of Hub Lock Axle Spacer into hole in Lower Hub Lock Assembly and hold in place over 3/4" nut. Place 1/4" spacer (item 21) between the Lower Hub Lock Assembly plate and the Hub Lock Axle Plate Adapter, aligning holes. Slide 1/4" x 1-3/4" bolt (item 10) through plate of Lower Hub Lock Assembly, 1/4" spacer (item 21) and Axle Plate Adapter (item 14). Attach nut provided with washer to 1/4" bolt as shown and tighten.

Hub Lock Installation

Set-up and Adjustment

7. Remove 2 screws (item 4) from clamp assembly. Wrap clamps around push handle back tube, reinsert screws, and leave them loose. Slide to desired position and tighten securely.
8. Attach new wheel and axle through axle sleeve.
9. Repeat steps 1-8 on opposite side.

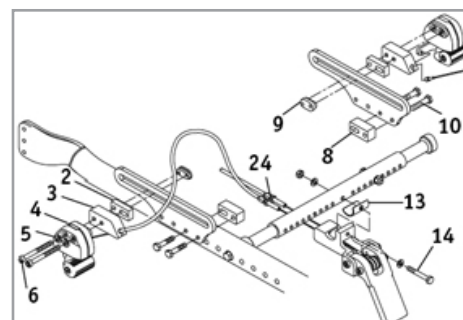
ATTENDANT WHEEL LOCK INSTALLATION

1. Remove wheels from chair.
2. Attach Mount bracket/cable assembly to rear base strut tube using 1/4" x 1-3/4" bolt (item 14) with nut and washers, being sure to go through spacer tube (item 13) provided as shown.
3. If push-to-lock wheel locks are already installed, remove the push-to-lock assemblies by removing the M6 bolts (item 6) and adjustment nuts (item 9), leaving the slide plates mounted on the chair, and go to step 5. If there are no wheel locks mounted already, proceed with step 4.
4. Attach Wheel Lock Slide Plates to Roller Assembly on each side of the chair using 1/4" x 1-1/4" bolts (item 10) and Mounting Block (item 8). If axle of rear wheel is above the base frame tube, orient the slide plates upward. If the axle of the rear wheel is below the base frame tube, orient the plates downward.
5. Attach the left and right Remote Lock Assemblies (item 4) to the slide plates using the Attendant Wheel Lock Spacer Blocks (item 3), M6 x 60 screws (item 6), lock washers (item 5), and Wheel Lock Adjustment Nuts (item 9). (If dual post armrests are installed, you may need to use the additional spacer block (item 2) and M6 x 70 bolts). The adjustment nuts marked "R" and "L" are to be mounted to the right and left side of the chair respectively. The smaller protruding oval shape on the nuts should slide inside of the slot in the slide plate. Do not tighten bolts. Slide the Remote Lock Assemblies as far forward as they will go.
6. Re-install wheels and slide the Remote Lock Assemblies rearward until the friction block of the remote lock is approximately 1/4" away from the Tyre when unlocked. If the lock cannot be adjusted to this position, you will need to adjust the position of the slide plate until it can be adjusted as stated. Once remote locks are properly positioned, tighten bolts.
7. Attach cables to Remote Lock Assemblies by sliding the stud at the end of the cable into the slot in the Remote Lock Assembly. Pull back on the cable sheath and slide the end of the cable sheath into the pocket on the Attendant Wheel Lock Spacer Block (item 3) as shown. Be sure that the cables are routed away from all moving parts. Attach the cables into place on the chair with zip-ties provided.
8. Adjust the cable adjustment screws (item 24) as necessary to ensure that the wheel locks can get to their fully extended position when the pedal is engaged. Position of the Remote Lock Assembly may need to be fine tuned to optimize the force needed to engage the pedal, depending on Tyre type. If excessive force is needed to engage the pedal, cable life may be reduced. Once proper adjustment is attained, lock cable adjustment screw in position by tightening jam nuts against mounting block.

CHECK-OUT

Once the wheelchair is assembled and adjusted, it should roll smoothly and easily. All accessories should also perform smoothly. If you have any problems, follow these procedures:

1. Review assembly and adjustment sections to make sure the chair was properly prepared.
2. Review troubleshooting guide.
3. If your problem persists, contact your authorized supplier. If you still have a problem after contacting your authorized supplier, contact Sunrise customer service. See the introduction page for details on how to contact your authorized supplier or Sunrise customer service.

Attendant Wheel Lock Installation

Battery Tray

Please Refer To Figures A and B Overleaf.

WARNINGS!

- Read the User's Manual before installing the battery tray components onto the wheelchair.
- Never use this wheelchair as a seat in a motor vehicle unless all components are removed from the battery tray, and properly secured separately in the vehicle.
- Always contact Technical Service before installing or removing the battery tray components to obtain the correct assembly locations of the strut tubes, roller carriage assembly and caster brackets.
- Do not install the battery tray components onto wheelchairs when the user weighs more than 150lbs.

NOTE: This option is only compatible with wheelchairs having a minimum floor-to-seat height of 15".

NOTE: The battery tray components are not compatible with the Attendant Wheel Lock option.

HOW TO MAKE WHEELCHAIR WIDTH ADJUSTMENTS

NOTE: The (2) strut tubes supplied with the battery tray kit create one unique wheelchair width.

Please contact Sunrise Medical to acquire strut tubes for other wheelchair widths.

1. Loosen and remove the bolts (A) from both ends of the two battery tray strut tube mounts (G)
2. Loosen bolts from both ends of the front strut tube.
3. Remove the battery tray.
4. Adjust the width of the chair.
5. Loosen and remove nuts and bolts (D) from the battery tray mounting brackets (E) on both strut tubes (F).
6. Remove Bolts and nuts (B) and replace the old battery tray strut tubes with the required size to match the desired width of the chair.
7. Insert the strut tubes (F) back into the battery tray mounting brackets (E), replacing bolts and nuts (D).
8. Fasten strut tube mounts (G) onto new battery tray strut tubes, replace and tighten the bolts (B). Heads of bolts should be on top.
9. Put the battery tray with the strut tubes back into place, replace and tighten bolts (A) into both ends of all strut tubes.

NOTE: Use a torque setting of 96 in-lb when tightening struts, carriages and axle plates.

Battery Tray

HOW TO ADD THE BATTERY TRAY KIT

1. Loosen and remove the bolts (A) from the rear strut tube.
The existing rear strut tube will not be used in the battery tray setup.
2. Relocation of carriage (when needed, consult customer service)
 - a. Loosen and remove rear bolt (i) from the carriage on each side.
 - b. Loosen and remove front bolt (ii) from the carriage on each side.
 - c. Set the carriage at desired location
 - d. Replace and tighten bolts
3. Relocation of Axle Plates (when needed, consult customer service)
 - a. Loosen and remove front and rear bolts from the axle plate on each side, noting the orientation and holes used on the axle plate.
 - b. Set the axle plate at new location
 - c. Replace and tighten bolts. Be sure to use the same orientation and holes in the axle plate as were being used previously, unless a seat height adjustment is being made.
4. Loosen bolts from both ends of the front strut tube.
5. Place Battery Tray with new middle and rear battery tray struts in proper locations (consult customer service for proper locations).
6. Replace and tighten bolts (A) into front strut tube and middle and rear strut tube mounts (G).

NOTE: Use a torque setting of 96 in-lb when tightening struts, carriages and axle plates.

HOW TO REMOVE BATTERY TRAY COMPONENTS

Always contact Technical Service before installing or removing the battery tray components to obtain the correct assembly locations of the strut tubes, roller carriage assembly and caster brackets.

1. Loosen and remove the bolts (A) from the middle and rear Battery Tray Strut Tubes. Relocation of carriage (when needed, consult customer service)
 - a. Loosen and remove rear bolt (i) from the carriage on each side.
 - b. Loosen and remove front bolt (ii) from the carriage on each side.
 - c. Set the carriage at desired location
 - d. Replace and tighten bolts
3. Relocation of Axle Plates (when needed, consult customer service)
 - a. Loosen and remove front and rear bolts from the axle plate on each side, noting the orientation and holes used on the axle plate.
 - b. Set the axle plate at new location
 - c. Replace and tighten bolts. Be sure to use the same orientation and holes in the axle plate as were being used previously, unless a seat height adjustment is being made.
4. Install Base Frame Strut (standard strut, not battery tray strut) into proper position (consult customer service for proper rear strut size and position), and tighten all strut bolts (A).

NOTE: Use a torque setting of 96 in-lb when tightening struts, carriages and axle plates.

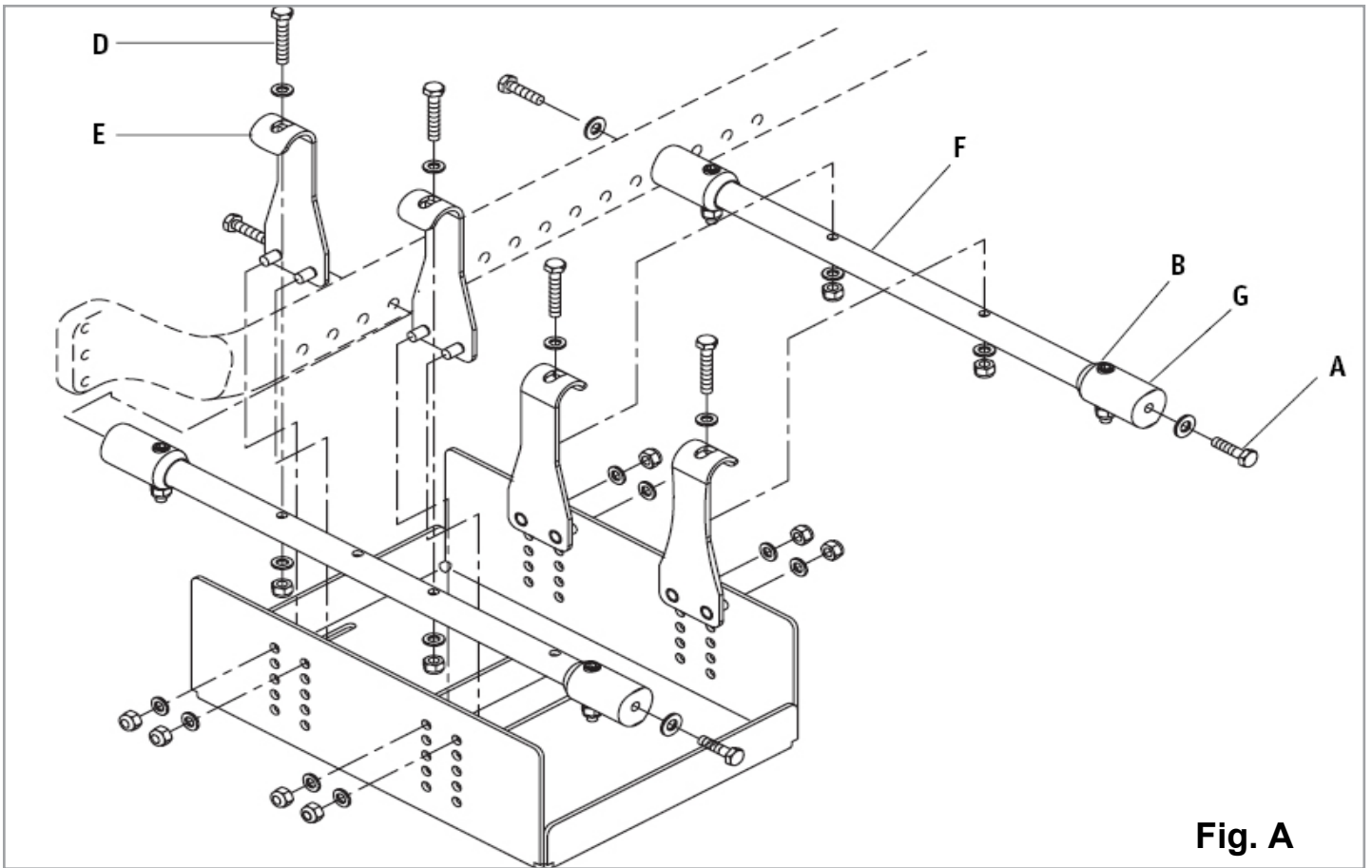
Fig. A

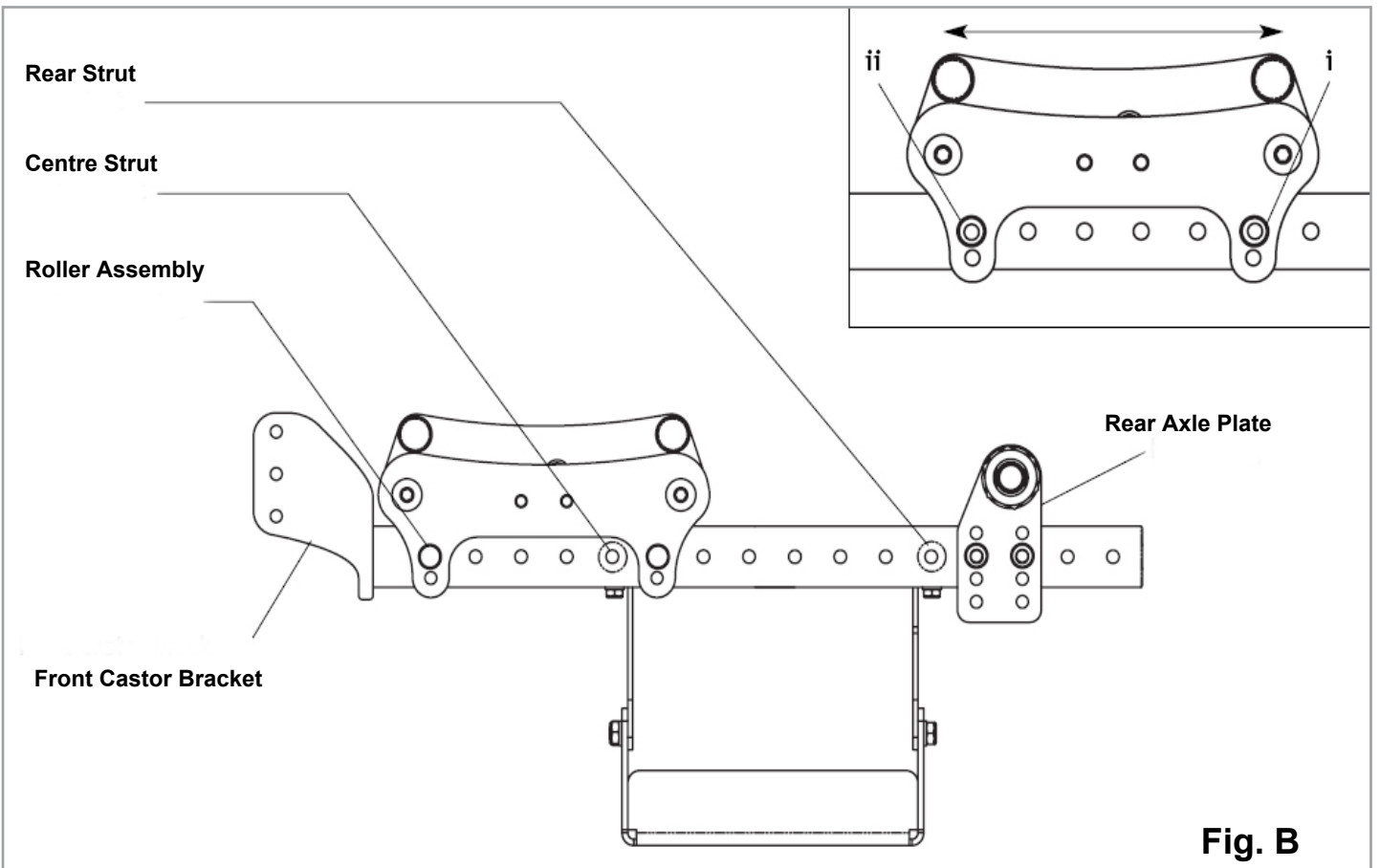
Fig. B

Positioning Belt

DANGER

- Positioning belts are fitted as positional aids, and are not suitable as transportation restraints.
- DO NOT over tighten positioning belt.
- Always make sure that the lap belt is correctly secured and adjusted prior to use. Too loose a strap could cause the user to slip down and risk suffocation or cause serious injury.
- The positioning belt must only be fitted by an approved Sunrise Medical dealer/agent.
- The positioning belt should only be adjusted by a professional, or a Sunrise Medical approved dealer/agent.
- The positioning belt must be checked on a daily basis to ensure it is adjusted correctly (see step 6) and is free from any obstruction or adverse wear.
- Sunrise Medical does not encourage the transportation of any person in a vehicle using this positioning belt as a method of restraint.

Positioning Belt Kit (Fig.4.33 - 4.37).

The positioning belt kit contains:

One positioning belt (A)
Three tri glide buckles (B)
Two cable ties (C)

1. To assemble the positioning belt, take the positioning belt strap and thread through the tri-glide. Repeat on both ends of positioning belt.
2. To fit the positioning belt to the chair, take the positioning belt strap and wrap around the back post.
3. Thread the positioning belt back through the tri-glide to form a loop around the back post. Thread positioning strap through tri-glide as shown.
4. To secure the positioning belt to the chair. Take one of the cable ties supplied and thread through the positioning belt strap. Then wrap around the frame and fasten together. DO NOT over tighten positioning belt. Repeat steps 2-4 with both ends of positioning belt.
5. To adjust the positioning belt use the tri-glide buckles and the fastening buckle.
6. The positioning belt should be adjusted so the fastening buckle is situated in the middle of the stomach with the straps at an angle of around 45°.
7. The positioning belt should be adjusted so a hand can be placed between the user and the belt.

WARNING!

- Check lap belt and securing components at regular intervals for any sign of frays or damage.
- Replace if necessary.
- The lap belt should be adjusted to suit the end user as detailed opposite.
- Sunrise Medical recommends that the length and fit of the belt be checked on a regular basis to reduce the risk of the end user inadvertently re-adjusting the belt to an excessive length.

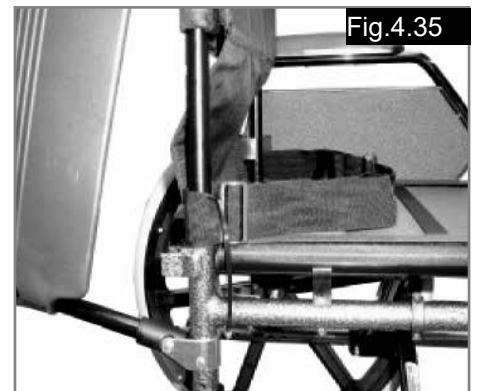
Fig.4.35

Fig.4.36

Fig.4.33

Fig.4.34

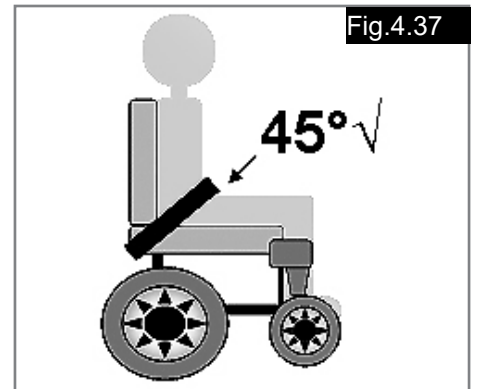
Fig.4.37

Transportability

DANGER!

There is a risk of serious injury or death if this advice is ignored!

Transportation of your wheelchair within a vehicle:

A wheelchair secured in a vehicle will not provide the equivalent level of safety and security as a vehicle seating system. It is always recommended that the user transfers to the vehicle seating. It is recognised that this is not always practical for the user to be transferred and in these circumstances, where the user must be transported whilst in the wheelchair, the following advice must be followed:

1. Confirm that the vehicle is suitably equipped to transport a passenger in a wheelchair, and ensure the method of access/egress is suitable for your wheelchair type. The vehicle should have the floor strength to take the combined weight of the user, the wheelchair and accessories.
2. Sufficient space should be available around the wheelchair to enable clear access to attach, tighten and release the wheelchair and occupant tie down restraints and safety belts.
3. The occupied wheelchair must be located in a forward facing position and secured by the wheelchair tie down and occupant restraint straps (WTORS tie downs meeting the requirements of ISO 10542 or SAE J2249) in accordance with the WTORS manufacturer's instructions.
4. The wheelchair's use in other positions within a vehicle has not been tested e.g. transportation in a side facing position must not be carried out under any circumstances (Fig. A).
5. The wheelchair should be secured by a Tie Down Restraint system, conforming to ISO 10542 or SAE J2249 with non-adjustable front straps and adjustable rear straps, which typically use Karabiner clips/S hooks and tongue and buckle fittings. These restraints generally comprise of 4 individual straps that are attached to each corner of the wheelchair.
6. The tie-down restraints should be fitted to the main frame of the wheelchair as indicated in the diagram on the following page, and not to any attachments or accessories, e.g. not around the spokes of wheels, brakes or footrests.
7. The tie-down restraints should be attached as close as possible at an angle of 45 degrees and tightened securely in accordance with the manufacturer's instructions.

8. Alterations or substitutions must not be made to the wheelchair securement points or to structural and frame or components without consulting the manufacturer. Failure to do so will invalidate the ability of a Sunrise Medical wheelchair to be transported within a vehicle.

9. Both pelvic and upper torso restraint belts must be used to restrain the occupant to reduce the possibility of head and chest impacts with the vehicle components and serious risk of injury to the user and other vehicle occupants. (Fig B) The upper torso restraint belt should be mounted to the vehicle "B" pillar - failure to do so will increase the risk of serious abdominal injuries to the user.

10. A head restraint suitable for transportation (see label on headrest) must be fitted and suitably positioned at all times during transportation.

11. Postural supports (lap straps, lap belts) should not be used or relied on for occupant restraint in a moving vehicle unless they are labelled as meeting the requirements specified in ISO 7176-19:2001 or SAE J2249.

12. The safety of the user during transportation depends upon the diligence of the person securing the tie-down restraints and they should have received appropriate instructions and/or training in their use.

13. Wherever possible remove and stow safely away from the wheelchair all auxiliary equipment, for example: Crutches, Loose cushions and Tray Tables.

14. Articulating/elevating leg rest should not be used in the elevated position when the wheelchair and user are being transported and the wheelchair is restrained using Wheelchair Transport and Occupant Restraints.

15. Reclining backrests should be returned to an upright position.

16. The manual brakes must be firmly applied.

17. Restraints should be mounted to the vehicle "B" pillar and should not be held away from the body by wheelchair components such as armrest or wheels.

Transportability Cont..

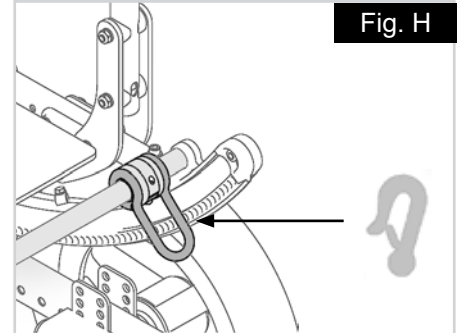
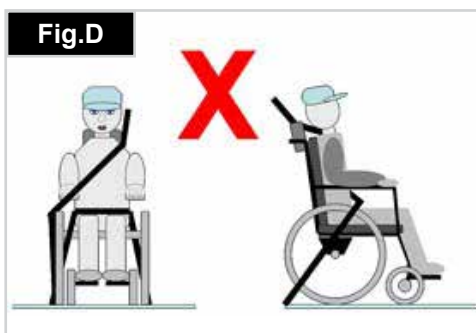
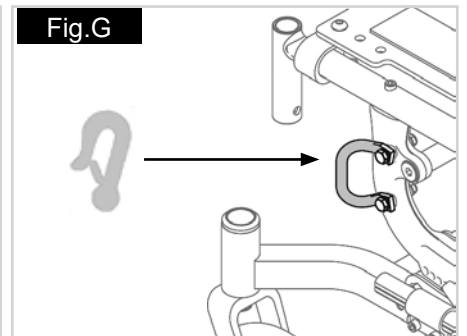
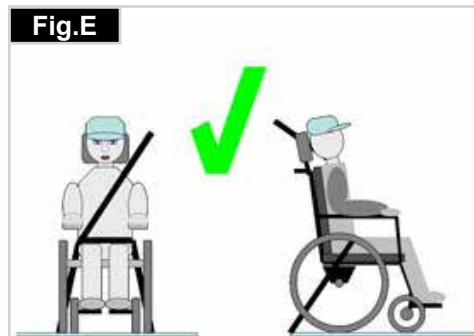
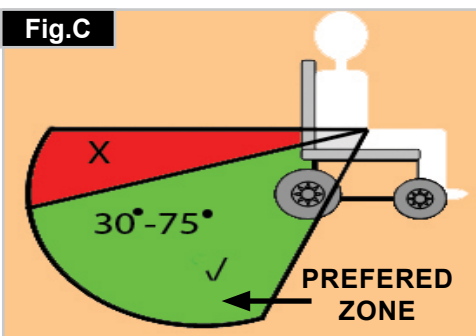
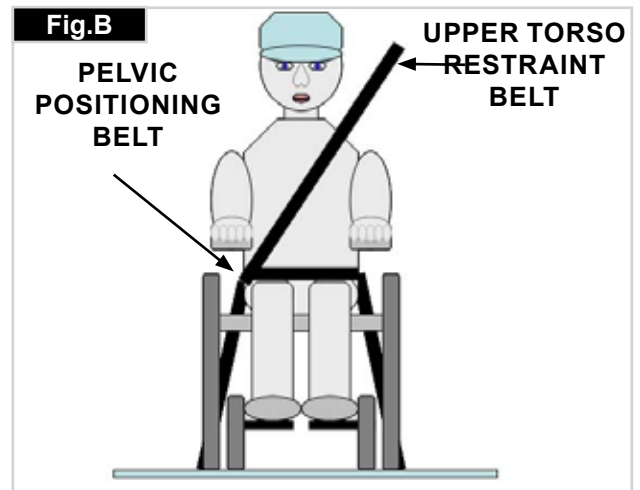
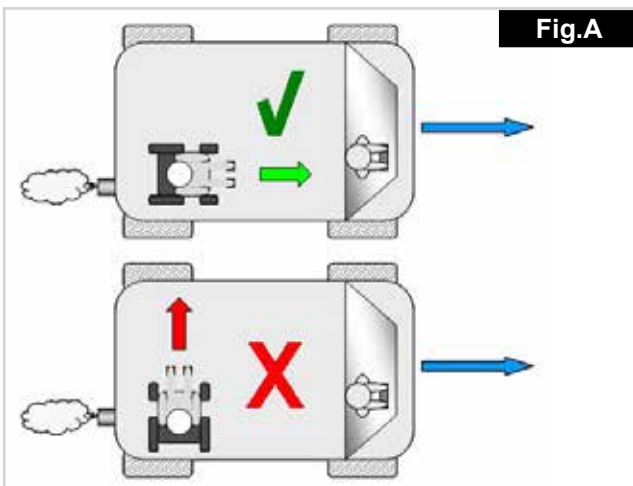
Occupant Restraints Instruction:

1. The pelvic restraint belt must be worn low across the front of the pelvis so that the angle of the pelvic belt is within the preferred zone of 30 to 75 degrees to the horizontal. A steeper (greater) angle within the preferred zone is desirable i.e. closer to, but never exceeding 75degrees. (Fig C)

2. The upper torso restraint belt must fit over the shoulder and across the chest as illustrated Fig d and e
 Restraint belts must be adjusted as tightly as possible consistent with user comfort.
 Restraint belt webbing must not be twisted when in use.
 The upper torso restraint belt must fit over the shoulder and across the shoulder as illustrated in Fig D and E.

3. The attachment points to the chair are the outer front side frames just above the castor, and the rear frame. The straps are fitted into the tie down brackets, (See Figs G - H)

4. The tie down symbol (Fig F) on the wheelchair frame indicates the position of the wheelchair restraint straps. The straps are then tensioned after the front straps have been fitted to secure the wheelchair.

Troubleshooting

Functional check

Once the wheelchair is assembled and adjusted, it should roll smoothly and easily. All accessories should also perform correctly.

If you have any problems, follow these procedures:

- Review assembly and adjustment sections to make sure the chair was properly prepared.
- Review troubleshooting guide.

You will need to adjust your chair from time to time for best performance, especially if you alter the original settings. The chart below gives you initial solutions.

Work from the left of the chart to the right, if needed, you may need to look farther to find the best solution for a specific problem.

- To keep track of your progress, make only ONE change at a time.
- If your problem persists, contact your authorized supplier. If you still have a problem after contacting your authorized supplier, contact Sunrise Medical customer service.
- See the introduction page for details on how to contact your authorized supplier or Sunrise Medical customer service.

Tyres and mounting

With pneumatic tyres make sure that you maintain the correct tyre pressure, as this can have an effect on wheelchair performance. If the tyre pressure is too low, rolling resistance will increase, requiring more effort to move the chair forward. Low tyre pressure also has a negative impact on manoeuvrability. If the tyre pressure is too high, the tyre could burst. The correct pressure for a given tyre is printed on the surface of the tyre itself.

Tyres can be mounted the same way as an ordinary bicycle tyre. Before installing a new inner tube, you should always make sure that the base of the rim and the interior of the tyre are free of foreign objects. Check the pressure after mounting or repairing a tyre. It is critical to your safety and to the wheelchair's performance that regulation air pressure be maintained and that tyres be in good condition.

	CHECK ALL TYRE PRESSURES	CHECK ALL LEFT AND RIGHT TYRE PRESSURES ARE EQUAL	CHECK ALL FASTENERS ARE TIGHT	USE A DRY LUBRICANT BETWEEN ALL FRAME CONNECTIONS AND PARTS.	CHECK CHAIR SETUP WITH YOUR DEALER
VEERS TO LEFT					
VEERS TO RIGHT					
CHAIR FEELS SLACK					
CHAIR IS HARD TO TURN					
CASTOR WOBBLE					
SQUEAKS AND RATTLES					
CHAIR TILTS FAST BUT HARD TO RETURN					
CHAIR RETURNS FAST BUT HARD TO TILT					

Maintenance and care

Safety Check:

As the user you will be the first person to notice any possible defects. We therefore recommend that before each use, you check the following:

- The tyre pressure is correct.
- The brakes work correctly.
- All removable parts are securely fastened (e.g. armrests, footrest hangers, quick-release axles etc).

If there is any damage/defect, please contact your uthorised dealer.

Maintenance:

- Check the tyre pressure once a week.
- Check all tyres for wear and damage at regular intervals, at least every 3 months. Change the tyres as soon as there is any sign of damage or wear.
- Check axles and axle sleeves every six months to make sure they are tight. Loose sleeves will damage the axle plate and will affect performance.
- Check the seat and back sling for wear and damage at regular intervals, at least annually. Change these items as soon as there is any sign of damage or wear.
- Check all frame and backrest components including fasteners for wear and damage at regular intervals, at least every 3 months. Change these items as soon as there is any sign of damage or wear.
- Check the brakes for wear and damage at regular intervals, at least every week. Check that they are working properly and are easy to use. Change the brakes as soon as there is any sign of damage or wear.

- Check to make sure all bolts are secure (see the section on torque) at regular intervals, at least every 3 months. All screws which are critical to using the wheelchair safely have safety nuts. Safety nuts should only be used once and should be replaced after use.

NOTE: If torque settings are given, then we strongly recommend that you use a torque measuring device, in order to check that you have tightened to the correct torque. Please use only mild household cleaners when your wheelchair is dirty. Use only soap and water when cleaning the seat upholstery and lap belt. Depending on the frequency and type of use, we recommend taking your wheelchair to your authorised dealer regularly, but at least within a year, to have it maintained by trained personnel.

CAUTION!

Sand, salt and sea water can damage the bearings of the front and rear wheels. Clean and dry the wheelchair carefully, after they have been exposed to these elements.

WARNING!

If you fail to heed these warnings your chair may fail and cause severe injury to the rider or others. Inspect and maintain this chair strictly per maintenance chart. If you detect a problem, make sure to service or repair the chair before use. At least once a year, have a complete inspection, safety check and service of your chair made by an authorised supplier.

ITEMS TO CHECK	WEEKLY	3-MONTHLY	6-MONTHLY	ANNUALLY
TYRE PRESSURES				
WHEEL LOCKS				
ALL FASTENERS				
ARM RESTS				
AXLES AND SLEEVES				
QUICK RELEASE AXLES				
WHEELS-SPOKES-TYRES				
CASTORS				
ANTI-TIPS				
FRAME				
ROLLERS				
TILT CABLE AND PLUNGER				
ROCKER ARM				
FULL DEALER SERVICE				

NORMAL CLEANING and checking

Proper maintenance will improve performance and extend the useful life of your chair.

- Clean your chair regularly. This will help you find loose or worn parts and make your chair easier to use.
- Repair or replace loose, worn, bent or damaged parts before using this chair.
- To protect your investment, have all major maintenance and repair work done by your authorized supplier.
- Clean the painted surfaces with mild soap at least once a month.
- Protect the paint with a coat of non-abrasive auto wax every three months.
- Clean around axles and moving parts WEEKLY with a slightly damp (not wet) cloth.
- Wipe off or blow away any fluff, dust or dirt on axles or moving parts.
- You do not need to grease or oil the chair.
- Wipe the rocker arm's lower surface WEEKLY with a slightly damp (not wet) cloth.
- Check the plunger on each side weekly, dismantle and clean if needed.

Hygiene when being reused:

When the chair is to be reused, it should be prepared carefully, and be wiped and treated with spray disinfectant on all surfaces which could come into contact with the user.

If you need to do this quickly, you must use a liquid, alcohol-based disinfectant suitable for medical products and devices. Please pay attention to the manufacturer's instructions of the disinfectant you are using.

STORAGE TIPS

Store your chair in a clean, dry area. If you fail to do so, parts may rust or corrode. Before using your chair, make sure it is in proper working order. Inspect and service all items as shown in sections 6 and 7. If stored for more than three months, have your chair inspected by an authorised supplier before use.

Name Plate

Nameplates

The nameplate is located on either the cross-tube assembly or the transverse frame tube, as well as on a label in the owner's manual. The nameplate indicates the exact model designation and other technical specifications. Please provide the following pieces of information whenever you have to order replacement parts or to file a claim:

Due to our policy of continuous improvement in the design of our wheelchairs, product specifications may vary slightly from the examples illustrated. All weight/dimensions and performance data are approximate and provided solely for guidance.

Sunrise Medical complies with the EU Medical Devices Directive 93/42/EEC

[SAMPLE]

QSR-017000

S/N: 

SUNRISE MEDICAL (US), LLC

MADE IN MEXICO

2842 BUSINESS PARK AVE

Year of Manufacture: YYYY

FRESNO, CA 93727-1328

Max User Weight: 75kg (165.5lbs)

Warranty

THIS IN NO WAY AFFECTS YOUR STATUTORY RIGHTS.

Warranty conditions:

- 1) The repair or replacement must be carried out by an authorised Sunrise Medical dealer/service agent.
- 2) To apply the warranty conditions, should your product require attention under these arrangements, notify the designated Sunrise Medical service agent immediately giving full information about the nature of the difficulty. Should you be operating the product away from the locality of the designated Sunrise Medical service agent work under the "Warranty Conditions" will be carried out by any other service agent designated by the manufacturer.
- 3) Should any part of the wheelchair require repair or replacement, as a result of a specific manufacturing or material defect, within 24 months, (5 years on frame and cross-brace), from the date on which the possession of the product was transferred to the original purchaser, and subject to it remaining within that ownership, the part or parts will be repaired or replaced completely free of charge if returned to the authorised service agent.

NOTE: This guarantee is not transferable.

- 4) Any repaired or replaced part will benefit from these arrangements for the balance of the warranty period applicable to the wheelchair.
- 5) Parts replaced after the original warranty has expired are covered for a further 24 months.
- 6) Items of a consumable nature will not generally be covered during the normal warranty period, unless such items have clearly suffered undue wear as a direct result of an original manufacturing defect. These items include amongst others upholstery, tyres, inner tubes, and similar parts.
- 7) The above warranty conditions apply to all product parts for models purchased at full retail price.
- 8) Under normal circumstances, no responsibility will be accepted where the product has required repair or replacement as a direct result of:
 - a) The product or part not having been maintained or serviced in accordance with the manufacturer's recommendations, as stated in the Owner's Manual and / or Service Manual. Failing to use only the specified original equipment parts.
 - b) The product or part having been damaged by neglect, accident or improper use.
 - c) The product or part having been altered from the manufacturer's specifications, or repairs having been attempted prior to the service agent being notified.
 - d) If the product has corrosion due to incorrect maintenance

Disposal / recycling of materials

NOTE: If the wheelchair has been supplied to you free of charge it may not belong to you. If it is no longer required follow any instructions given by the organisation issuing the wheelchair in order that it may be returned to them.

The following information describes the materials used in the wheelchair in relation to the disposal or re-cycling of the wheelchair and its packaging. Specific waste disposal or recycling regulations may be in force locally and these should be taken into consideration when disposal arrangements are made. (This may include the cleaning or de-contamination of the wheelchair before disposal).

Aluminium: Castor forks, wheels, side frames, armrests, frame, leg rests, push handles

Steel: Fasteners, QR axle

Plastic: Handgrips, tube plugs, castor wheels, footplates, arm pads and 12" wheel/tyre

Packaging: Low density polythene bag, cardboard box

Upholstery: Woven polyester with PVC coatings and expanded combustion modified foam.

Disposal or recycling should be done through a licensed agent or authorised place of disposal. Alternatively your wheelchair may be returned to your dealer for disposal.

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Sunrise Medical B.V.
Sunrise Medical HCM B.V.
Sunrise Medical S.A.S